WOUND HEALING TREATMENT

RI.MOS.

Disposable Medical Products

www.rimos.com
info@rimos.com

ENG

FOR VETERINARY USE
For pets and sports horses
NON FOOD PRODUCTION

Helps skin healing therapy
Hypermixvet stimulates skin healing in all cases of external injuries. The product is disliked by insects. It can be applied to the layers involved in the injury, on the mucosa such as mouth, gums, anus and in the ear duct. Controls the quantity and quality of exudates on the surface of the wound.

How to use
Apply Hypermixvet on the well-cleansed injured surface. The dried exudates must be removed by rubbing the injured surface lightly with gauze dipped in sterile water or hypertonic saline solution. Any scabs must always be removed either surgically or with a hypertonic saline solution, better with hyper-saturated solution. The wounds must be cleansed with sterile water or saline solution, if necessary using non-irritating soap (Marseilles soap), without using disinfectants especially with alcohol, iodine, chlorhexidine or hydrogen peroxide base. As far as possible, foreign bodies must be mechanically removed. Bandaging dogs and cats is at the discretion of the vet, even though it is best to bandage cats.

Use restrictions
The product is ineffective on necrotic tissue and in cases of skin hyperkeratosis (psoriasis, skin thickening, corns, etc.).
Contraindications
Prolonged product application can produce the momentary fall out of hair in the injured area.

41037, Mirandola (MO)
ITALY
phone: 39 0535 25755
fax: 39 0535 25509

Warnings
In many cases, eliminates the need for local antibiotic therapy or reduces systemic antibiotic therapy. Avoid using disinfectants as these negatively affect the healing process.
The product fears air and light (close the container after use and store in a dark place). It is not affected by heat up to around 40°C. The product solidifies at temperatures below 16° and this does not alter its properties. It can also be used in solid state and immediately returns to fluid state at body temperature.
In case of individual and/or allergic hypersensitivity to ingredients, interrupt application.
Do not use the soaked gauzes again after use. After use, dispose of the soaked gauzes and containers in accordance with the law. Keep out of reach of children.

Components
Extracts from the Neem tree, Hypericum Perforatum

Available packs
250ml glass bottle, oily formula.
100ml glass bottle, oily formula with spray nozzle.
30ml glass bottle, oily formula with dropper.
100 ml gel spray with spray nozzle.

SHAKE WELL BEFORE USE

The expiry date shown on the pack refers to the product in unopened and correctly stored pack.
REV. January 2011

Do not discard in the environment
Important: see instructions for use

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