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**Hysterosalpingography / sonohysterography uterine injector
with mandrel**

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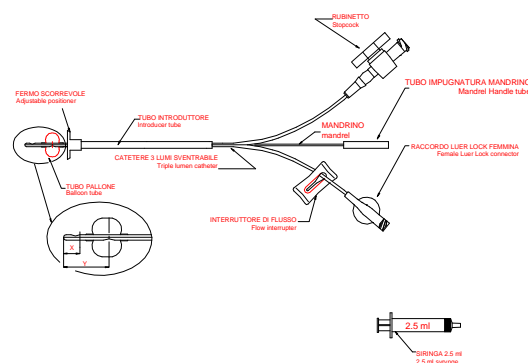
GB

Instructions for use

Pls notice: asymmetrical inflation of the balloon does not compromise the correct working of the device as once it is placed in the uterine cavity it assumes a symmetrical shape.

1. Expose the cervix using a speculum and cleanse it with disinfectant liquid.
2. Perform a hysteroscopy to measure the depth of the uterus.
3. Take INJECTOR PLUS out of sterile packaging and check balloon integrity (inflate and deflate with a syringe).
4. Lubricate the tip of the catheter and the balloon; insert the catheter gently into the uterine cavity through the cervical canal. When penetration is difficult it is possible to shape the catheter tip manually, thanks to the inner steel stylet (8). Caution: the catheter must be placed 1 cm underneath detected depth.
5. Gently inflate the balloon with 2 cc air by the stopcock and remove the stylet. **(Warning ! The Company is not responsible for any balloon bursting in case the maximum recommended inflation volume is exceeded).** The catheter is now safely placed into the uterine cavity.
6. Inject the contrast medium (or physiological solution in case of sonohysterography) through the second way to display the uterus and the Fallopian tubes.
7. At the end of the procedure deflate the balloon and remove the catheter.

Caution: to be used only after performing a hysteroscopy to avoid perforation of the uterine fundus. **The device does contain phthalates but it is not used on pregnant women or children. If accidentally used on pregnant women, the using time is so short that it cannot be dangerous for patients health.** Read the instruction before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose of after use according to the laws in force. For use only under trained medical staff.

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