**GINRAM®**

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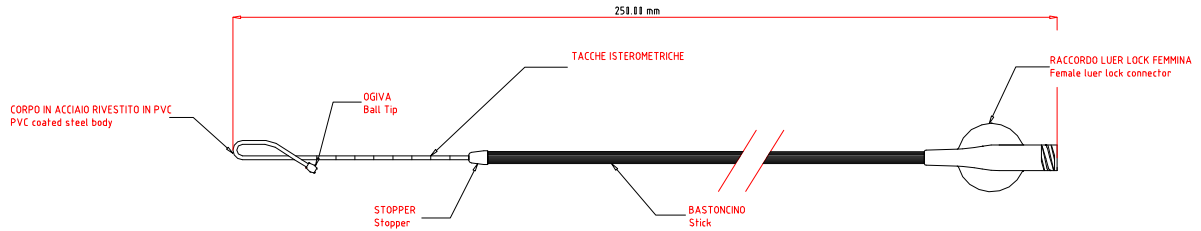
GB

### Instructions for use

Before using the device it is necessary to locate the position of the I.U.D. to be extracted with an ultrasound scan in the uterine cavity.

Accurately explore the vagina.

Accurately disinfect the portio.

- a) Carry out the hysteroscopy with appropriate straight hysteroscope, or curved hysteroscope in case of retroflexion or stressed antelexion (in this case it may be necessary to apply neck forceps on the portio).
- b) Cautiously insert the I.U.D. Extractor until marking the set hysteroscopic, impressing a delicate clockwise rotation on it: the gesture will ease the introduction, avoiding tissue traumas.
- c) Once near the bottom of the uterine, carry out another movement, this time anti-clockwise: the gesture will ease the hooking of the I.U.D.
- d) Cautiously retract the device until, once near the uterine external orifice, the marker wire or part of the rigid structure of the I.U.D., is visible.
- e) Cease the traction on the extractor!
- f) Use the forceps suitable for hooking the marker wire or a segment of the I.U.D., in this way completing the removal.

**Caution:** Read the instructions before use. Do not reuse. Use only if the package is not damaged. Use immediately after opening. For use only under trained medical staff.

**Do not use the device in case of: previous myomectomy interventions or repeated hysteroscopy, marked antelexion or retroflexion of the uterus, inflammation in progress.**

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