GINRAM[®]

GYNAECOLOGY AND OBSTETRICS
GYNÉCOLOGIE ET OBSTÉTRIQUE
GINECOLOGÍA Y OBSTETRICIA

FERTIRAM[®]

MAR (MEDICALLY ASSISTED REPRODUCTION)
PMA (PROCRÉATION MÉDICALEMENT ASSISTÉE)
RMA (REPRODUCCIÓN MÉDICAMENTE ASISTIDA)

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RI.MOS.

Medical Products

Visit **www.rimos.com** website
for a complete and constantly up-to-date overview
of RI.MOS. product ranges
as well as **of our latest news.**

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Cervical and Endocervical Sampling

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Embryo Transfer Transfert d'embryons / Transferencia de embriones

TRANSFER OMNI-L	55
TRANSFER OMNI-S	55
TRANSFER EASY	57

BUSINESS POLICY	59
POLITIQUE COMMERCIALE	60
POLÍTICA COMERCIAL	61

-  **RI.MOS.** is located in Mirandola (MO), an area known as **"the plastic valley"**, an important biomedical district acknowledged at European level.
-  **RI.MOS.** est située à Mirandola (MO), une zone connue sous le nom de **"plastic valley"**, un important district biomédical reconnu au niveau européen.
-  **RI.MOS.** se encuentra en Mirandola (MO), una zona conocida como **"the plastic valley"**, un importante distrito biomédico reconocido a nivel europeo.

COMPANY HISTORY

RI.MOS. was founded in 1988 as a manufacturer of disposable medical devices in the fields of **gynecology and assisted fertilization**. An important intuition, that immediately distinguishes the company from the others present in the biomedical district of Mirandola (Modena) - that since the early seventies was revolutionizing the Italian economic fabric.

Three years earlier the same partners had founded the company RAM, **the world's first manufacturer of single-use multi-injectors for mesotherapy**. The drive for innovation, therefore, characterizes the company throughout its evolutionary path.

For over thirty years RI.MOS. continues to improve its products, expanding its range with innovative lines of **herbal-based medical devices and functional cosmetics**. The plant-based line was born, in fact, from a revolutionary approach for the biomedical sector, which has been able to grasp the potential of plants (such as Neem and Hypericum) with multifunctional active principles, whose purpose is **to assist the treatment of all types of skin injury, in support not only to human medicine, but also to veterinary medicine**.

The company culture is based on **shared values of positive attitude, well-being of employees and customers** to whom the products are intended, **passion for the future, sustainability, work ethics**. RI.MOS. exports **Made in Italy** to over 90 countries around the world.

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🇬🇧 TECHNOLOGY AND CERTIFICATIONS

RI.MOS. works with qualified medical professionals and competent, up-to-date scientific researchers to develop its disposable medical devices. Research and design take place in-house by **ENKI**, a company that forms a group with RI.MOS. specialising in technological solutions and prototyping for the biomedical sector. The medical device manufacturing process involves highly experienced employees and suppliers. Purchasing the latest production technologies, RI.MOS. devices are **CE marked** and assembled by qualified personnel in **ISO 7 and ISO 8 cleanrooms** in compliance with an **ISO 13485 certified quality** management system.

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Laboratory of industrial research
 Art. no.14 of Ministerial Decree no.593 of 8/08/2000
 G.U. no.87 of 14/04/2007
 Ministry of Education,
 University and Research

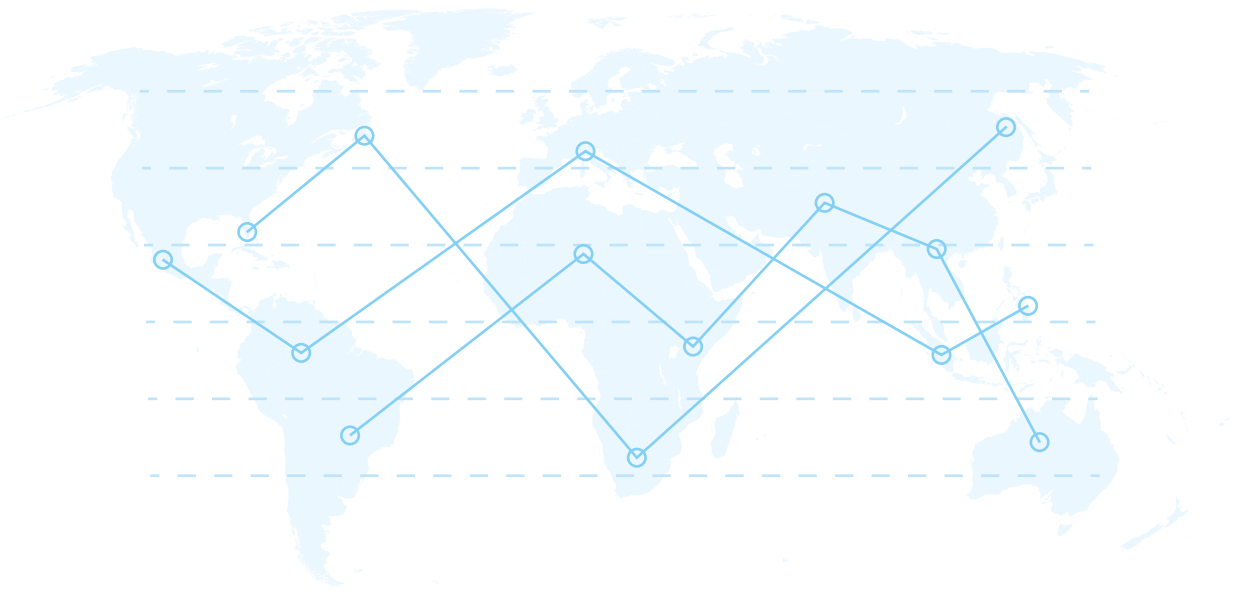
In 2007 RI.MOS. was accredited as a MIUR Laboratory (Ministry of University and Research).

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MARKETS

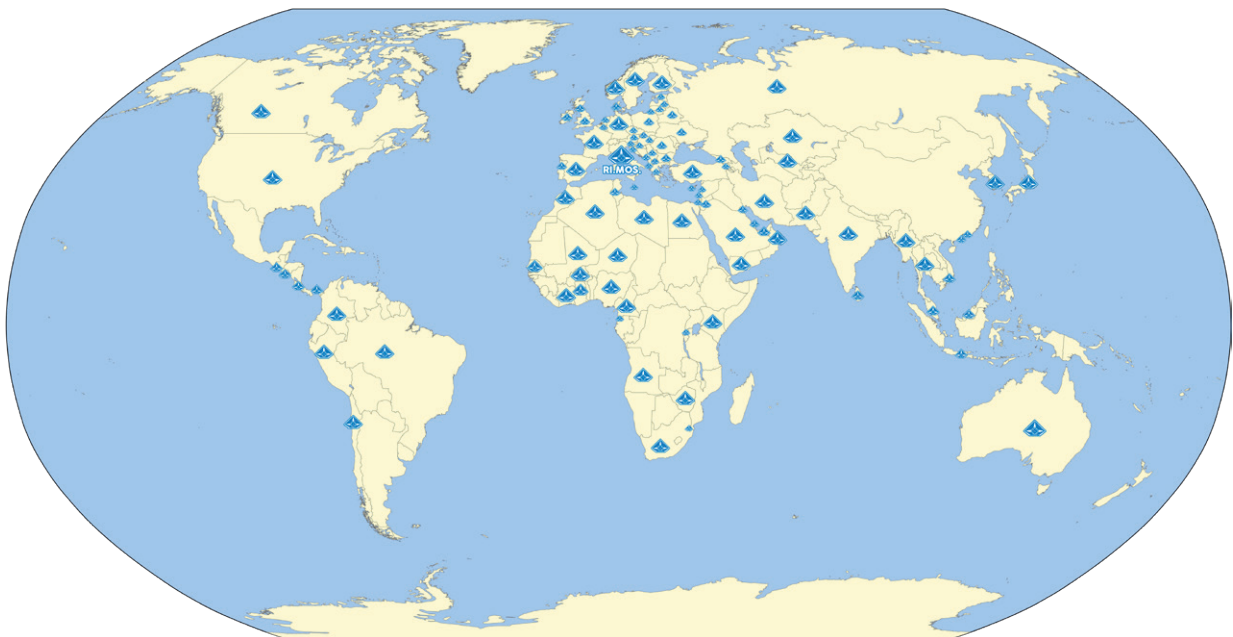
Domestic market involves retailers, local dealers and direct sales agents organized in a capillary network. Export quota is 55% of the total turnover with sales managed through distributors located in more than 90 countries.

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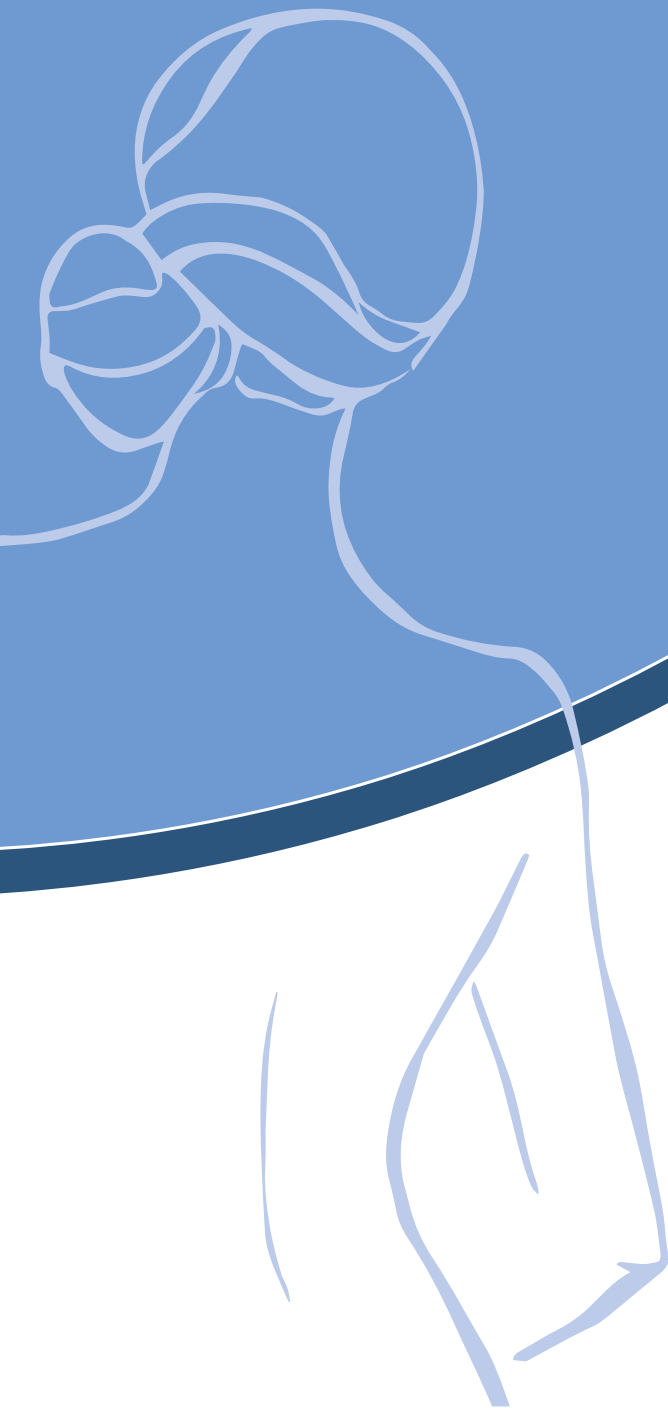
For the control or temporary reduction of the postpartum uterine bleeding when conservative management is warranted.

Catheter equipped with 2 balloons:

an external one (1) to be inflated with sterile saline solution (maximum inflation 60 ml) through the blue-colored "water" lumen (3), and an internal one (2) to be inflated with air through the transparent "air inflation" lumen (suggested inflation: 360 cc of air, equivalent to 6 pumping with the 60 ml syringe; maximum inflation 420 cc).

The special inflation valve (4) of the transparent lumen enables to maintain the syringe connected during the inflation process.

The red drainage lumen (5) must be connected to the fitted fluid collection bag (6) for the correct monitoring of the blood loss.

This specific design of the balloons enables the device to be very light, thus avoiding the risk of balloon slippage.

The inflation and deflation process is very fast and it is possible to reposition the catheter in case of misplacement.

Ease of use by the physician and total comfort for the patient.

The kit consists of catheter, 60 ml syringe and fluid collection bag.

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
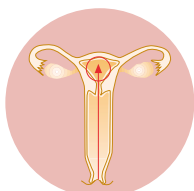
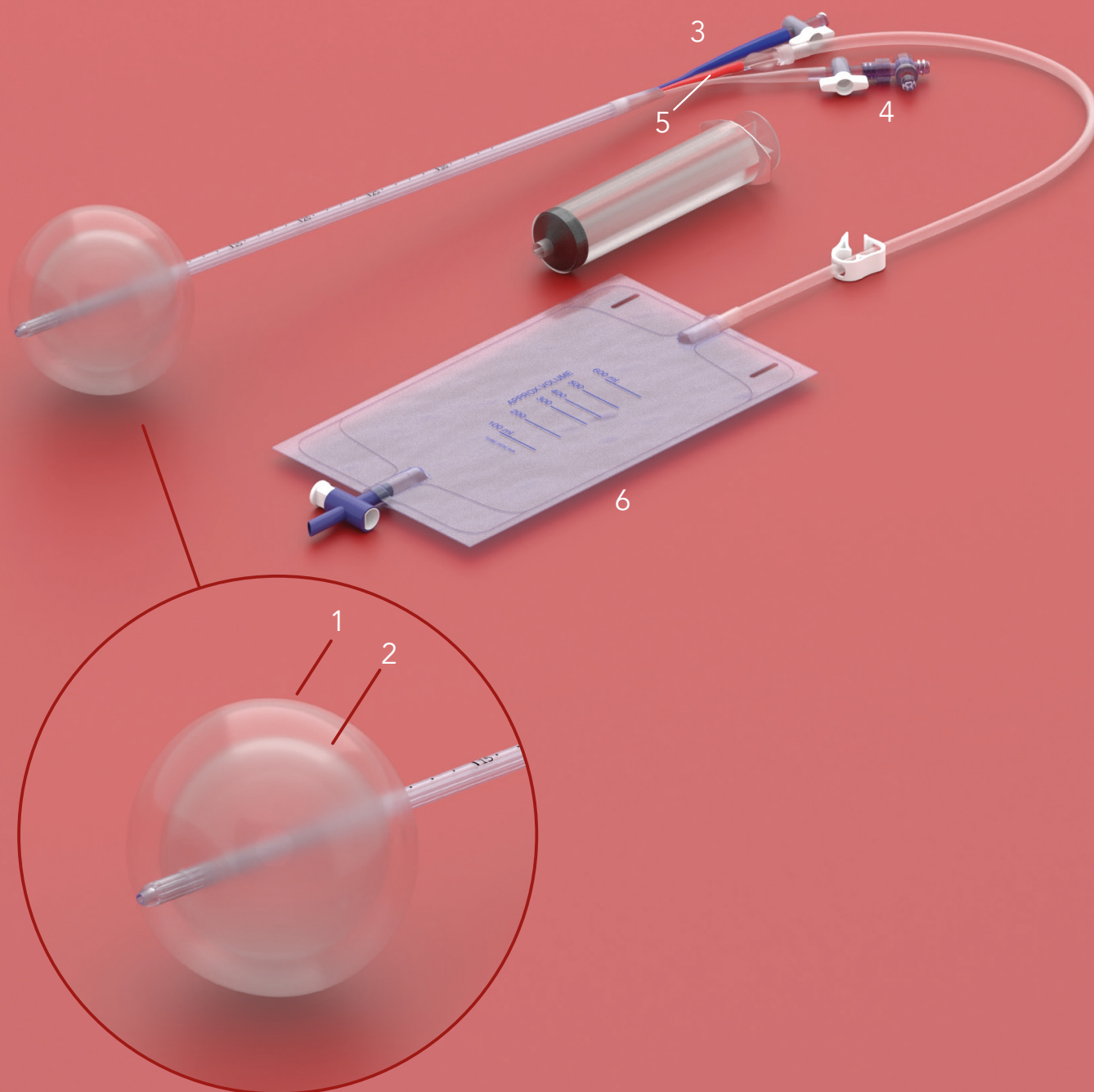
Vagistop

Device for prevention and treatment of vaginal bleeding in gynecology and obstetrics

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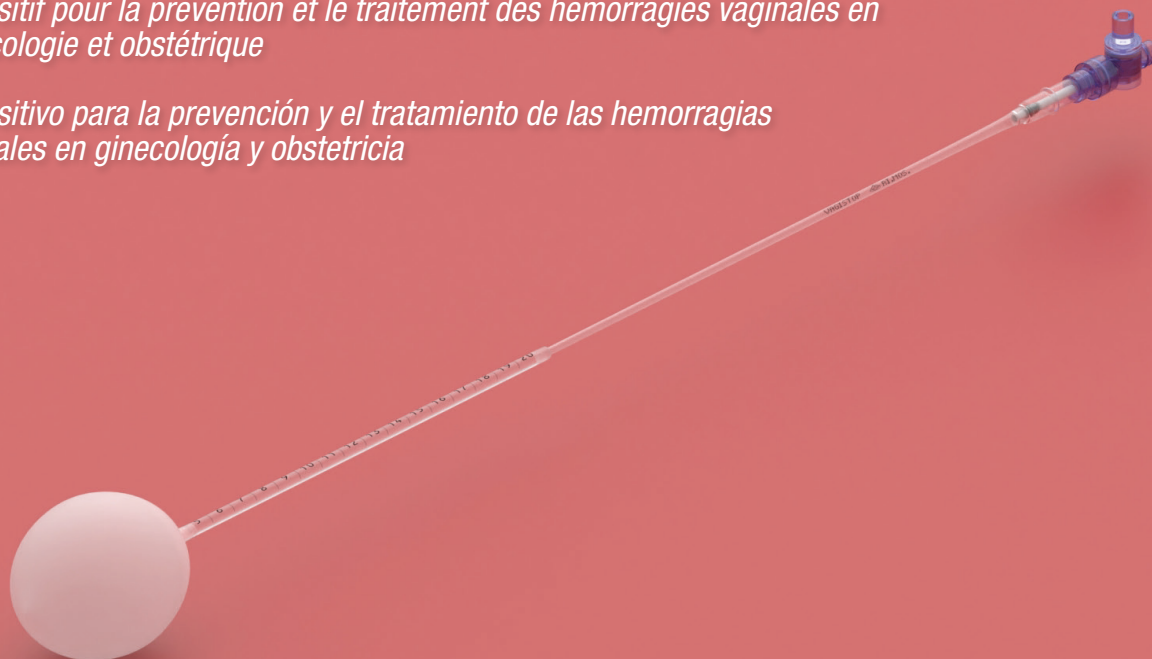• **Vagistop in Gynecology:** prevention and treatment of vaginal bleeding and hematoma after vaginal surgery or after trauma. It replaces the use of vaginal packing with gauzes. Air Inflation. Recommended inflation volumes for Gynecological use: from 60 to 180 ml depending on the vaginal size.

• **Vagistop in Obstetrics:** treatment of bleeding from postpartum vaginal tears and prevention of postpartum vaginal hematoma. It replaces the use of vaginal packing with gauzes. Air Inflation. Recommended inflation volumes for Obstetric use: from 240 to 480 ml depending on of vaginal size. When inflated to adhere to the vaginal walls Vagistop maintains an internal pressure of about 50 mmHg, therefore higher than the pelvic capillary pressure, which varies from 21 to 48 mmHg. Tested up to a maximum air inflation of 500 ml.

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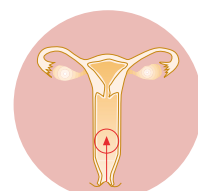
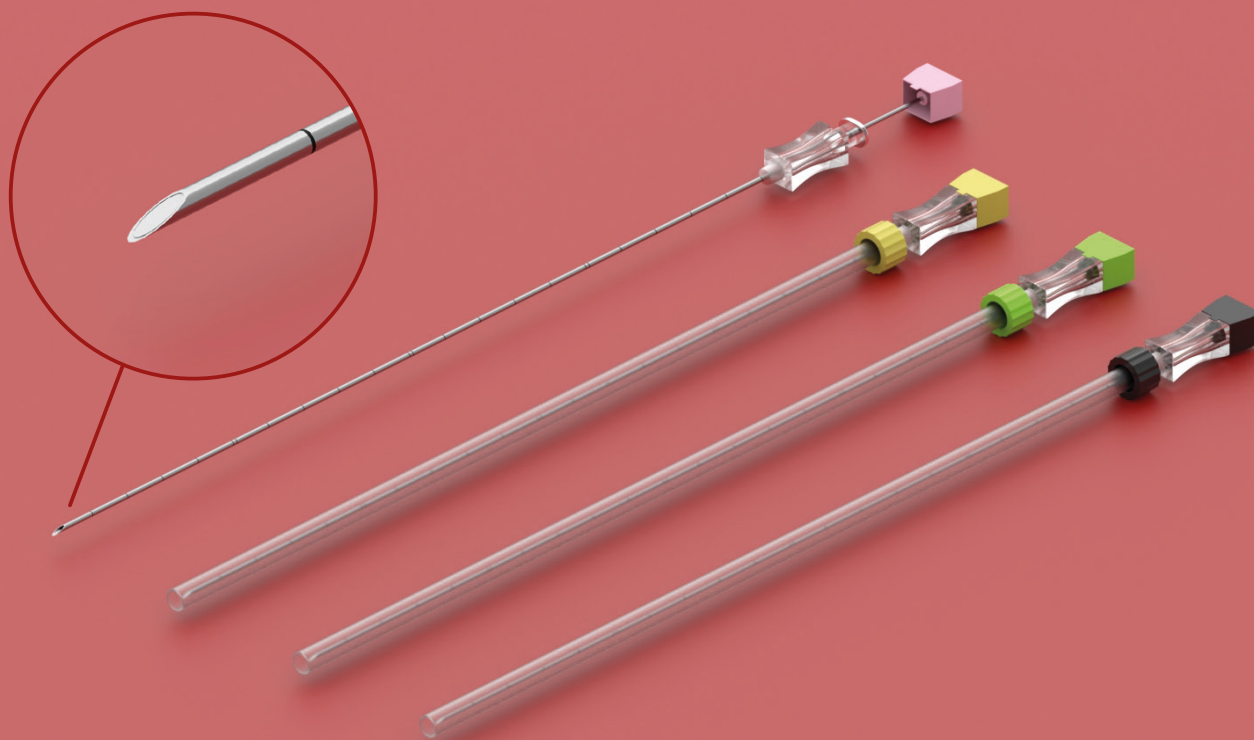
The Chiba model disposable needle is designed for cytological aspiration.

It features a cannula with centimetre markings, an echogenic treatment ensuring precise positioning under ultrasound guidance, and adheres to an internationally recognized colour code for diameter identification.

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
Amnioram

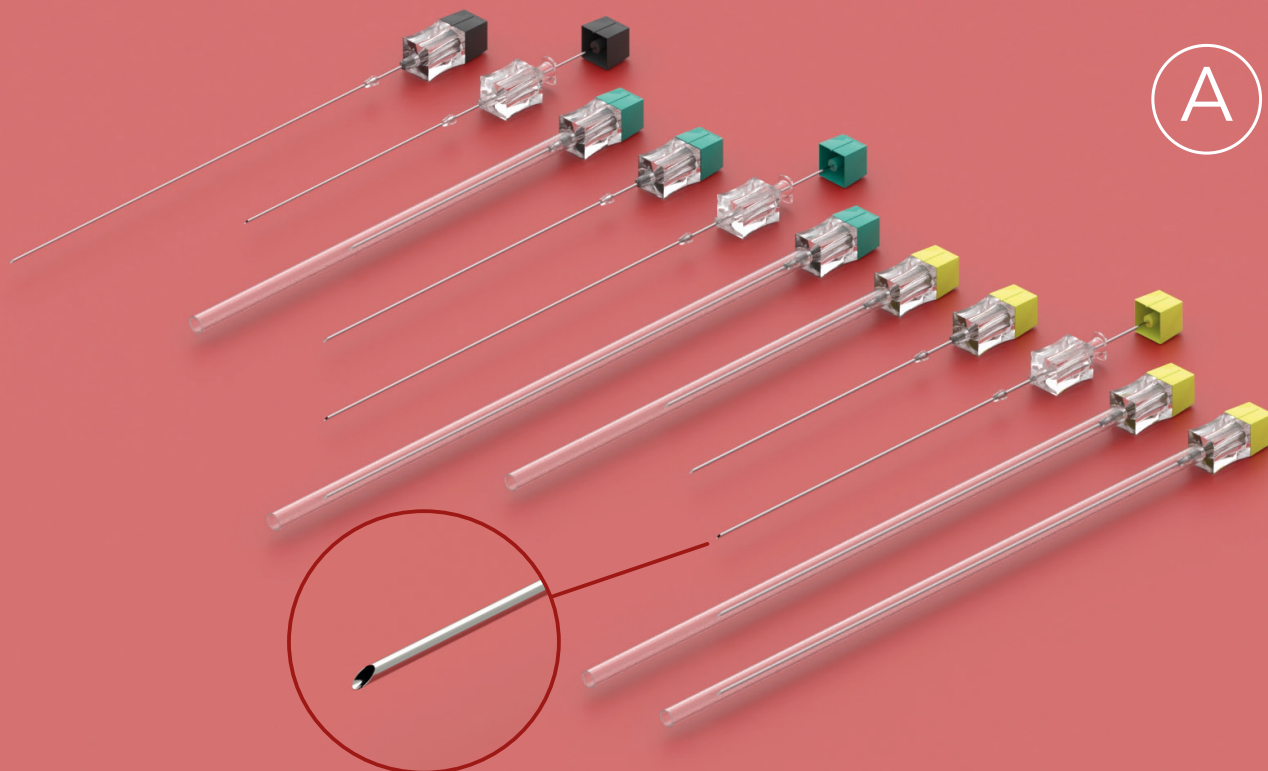Amniocentesis needles
Aiguille pour amniocentèse
Aguja para amniocentesis

For transabdominal sampling of amniotic fluid (usually performed between the 15th and 18th pregnancy week) for prenatal diagnosis of genetic disorders of the foetus.

It is also used for cytogenetic and biochemical studies.

The mandrel is provided with an ultrasound detectable tip allowing an echo-guided insertion.

The needle is provided with an adjustable stopper to determine insertion depth, and with a luer lock connector to collect the amniotic fluid through a syringe.

AMNIORAM (A) is available in 11 different sizes with no centimeter marking.

AMNIOCENT (B) is available in 4 different sizes.

The needles are centimeter-marked (1 cm resolution)* and has a double notch at 5 cm.

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
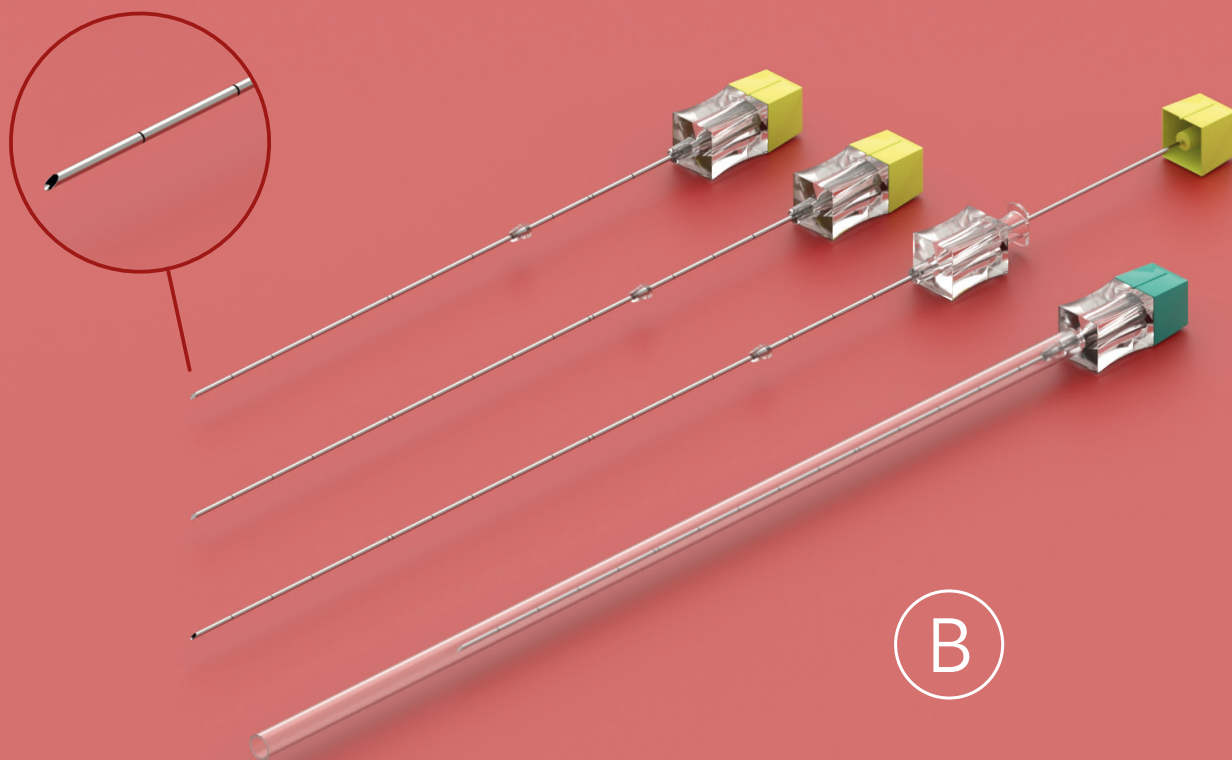
Amnioper

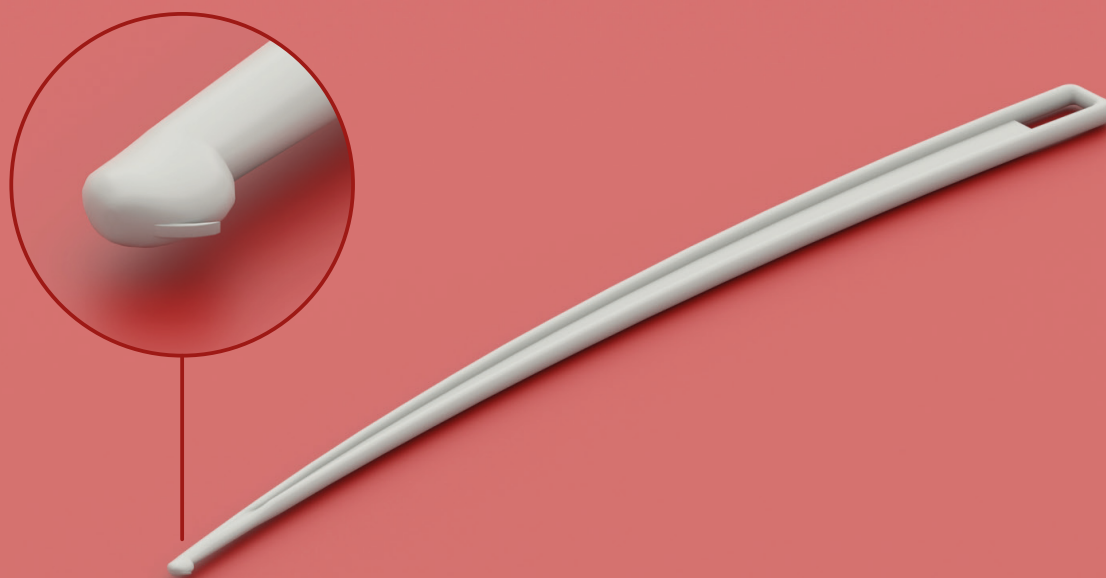Amniotic perforator
Perforateur amniotique
Perforador amniótico

For intentional rupture of the amniotic membranes during labour.
It is provided with a sharp and atraumatic hook allowing membrane snagging.
Its anatomic shape and grooved grip allow a safe and comfortable use.

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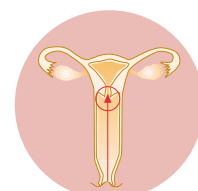
Ramclamp

Umbilical cord clamp
Clamp ombilical de Bar
Pinza para el cordón umbilical

Used to clamp the umbilical cord of the newborn before cutting. It is provided with a safety lock to avoid accidental opening. Once closed it can be opened with suitable forceps only. It is practical to use also with one hand.

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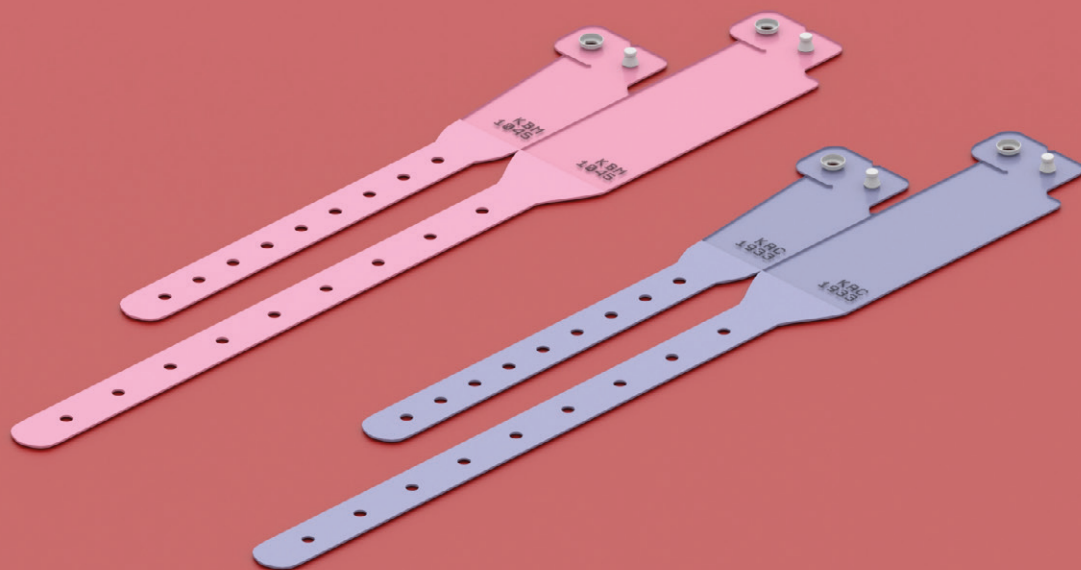
The mother/baby identification bands are tied on one side and provided with same number to avoid any identification error.

It is possible to write mother/baby personal data on the proper space.

24 cm long (mother) and 15.5 cm long (baby). Available in pink and light blue.

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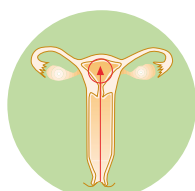
Intracurette RC

*Rigid curved intrauterine
suction curette*

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Used for curettage and evacuation of the endometrial contents during induced termination of pregnancy or after miscarriage. A frosted strip on the curette aids in visualizing insertion depth. Smooth edge tip. It can be connected directly to the suction tubing. Available in 6 different diameters.

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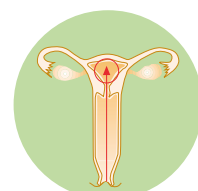
Intracurette SR

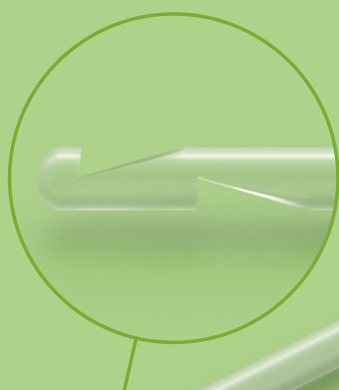
*Flexible intrauterine suction curette
"Karman" style*

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Intrauterine aspiration
Aspiration intra-utérine
Aspiración intrauterina

21

Used for curettage and evacuation of the endometrial contents during induced termination of pregnancy or after miscarriage. Flexibility of the material allows a gentle curettage. Available in 9 different colour-coded diameters for an easy identification. It can be connected directly to the suction tubing.

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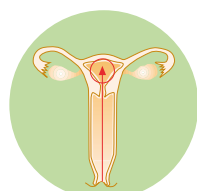
Manipolo Rovac

Used during voluntary pregnancy termination procedures or following miscarriage.

The rotating handle (1) is connected to a 2 meters tubing set (2) ending with a suction pump adapter.

Suction is regulated through a control valve that can be easily activated with one finger.

360° rotating handle allows a complete evacuation of the uterine cavity and prevents tubing twisting.

A reference mark placed on the handle indicates the exact orientation of the curette tip.

Tube thickness prevents any possible collapse and great transparency makes aspirated material visible.

It can be used with all suction pumps available on the market.

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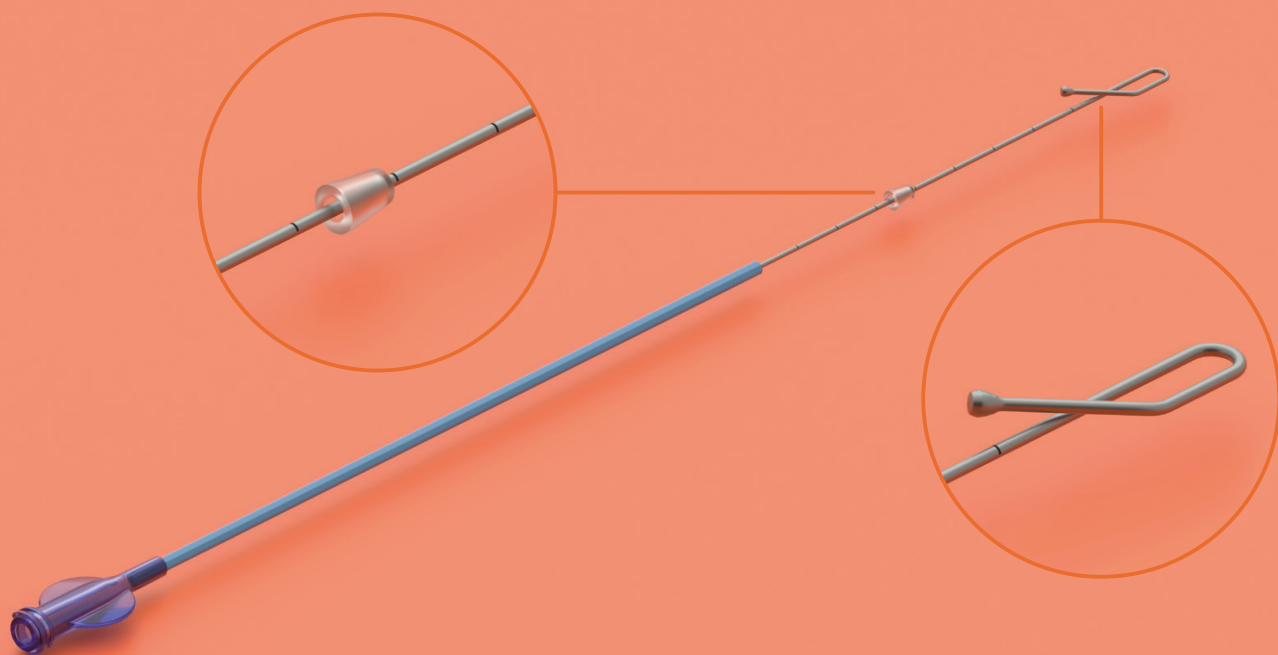
Designed for removing IUDs when the string is not visible or in presence of IUDs without strings.

The design of the tip makes the instrument easy to insert and allows simplified hooking of the IUD string or structure.

The ball at the end of the tip prevents tissue trauma.

The steel cannula has a 9 cm graduation marking (resolution 1 cm)* and is provided with a sliding positioner to determine the insertion depth.

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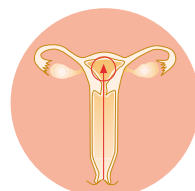
 IUD-S KIT is a ready-to-use kit allowing to perform the entire procedure of insertion, positioning and subsequent removal of the intra uterine device.

The kit includes:

- one hystrometer (RAMISTER) to assess uterus depth;
- one Pozzi Palmer forceps (TIRAPALLE) to widen uterus neck;
- one Kelly forcep (KELLY) to insert the IUD and place it in the correct position;
- long plastic scissors (U-CUT) allowing to cut the IUD string once the device has been positioned in the uterus.

Available also without scissors (IUD KIT).

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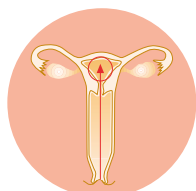
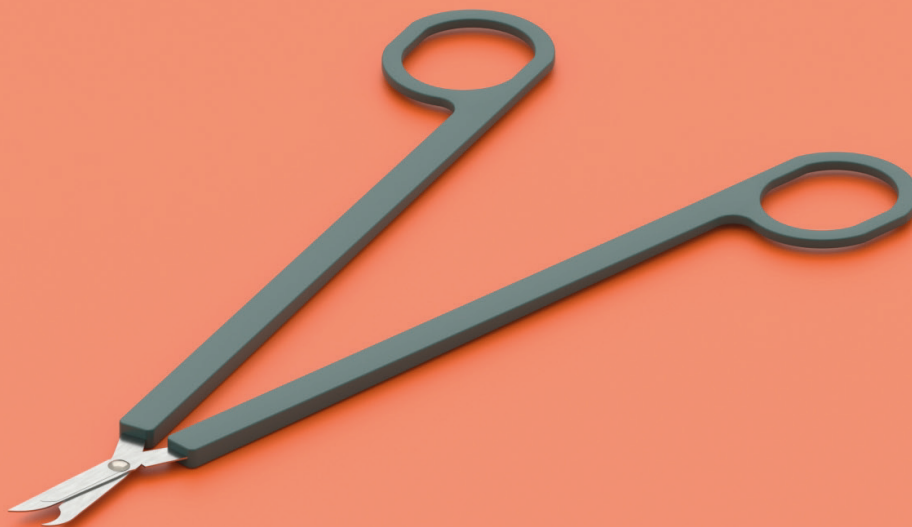
It is used for suture removal, general medical procedures and where it is necessary to cut strings, gauzes and similar.

U-CUT is a plastic disposable scissor whose main feature is the long and narrow shape; it is provided with stainless steel blades with accurate sharpening allowing a precise and safe cut.

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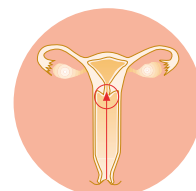
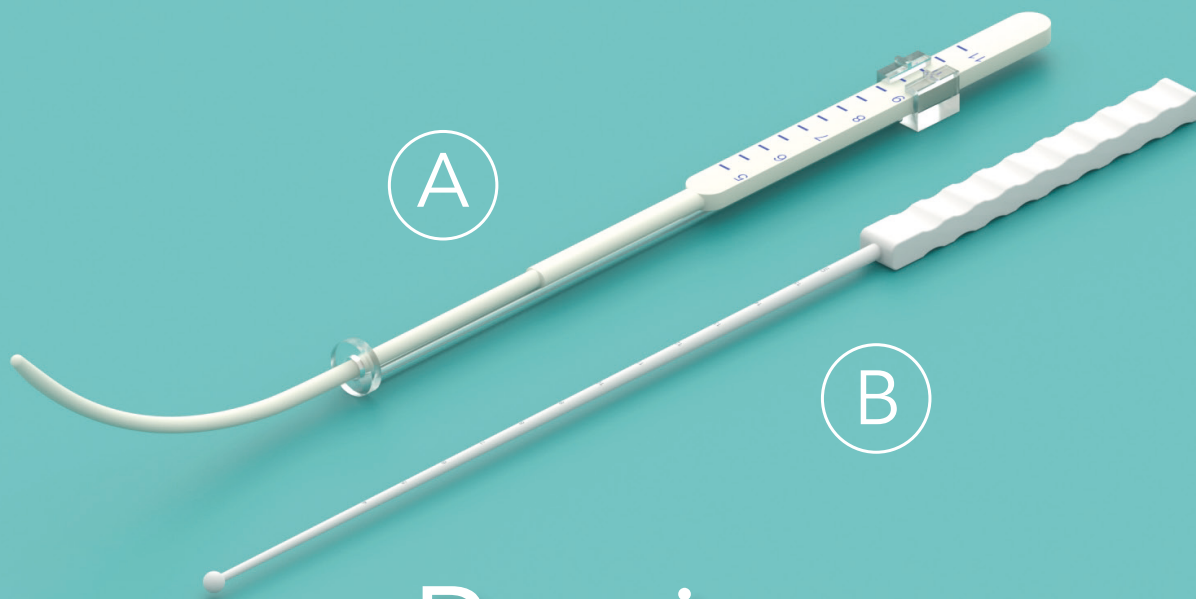
• **Ramister-C:** It provides an approximate measure of the depth of the uterus.

Curved and graduated, ranging from 5 to 11 cm (0.5 cm resolution)* with direct reading on the handle. It is easy to place, thanks to the adjustable positioner to be set on the cervix external orifice. Slightly conical configuration with decreasing diameter 4 to 3 mm.

• **Ramister:** It provides an approximate measure of the depth of the uterus.

Straight and graduated, ranging from 4 to 15 cm (1 cm resolution)*. Slightly conical (5 to 3 mm O.D.).

The spherical-tip allows a comfortable insertion through the cervical canal.

Its malleable plastic allows manual bending of the instrument when necessary.

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
*COD. 720108: available in boxes of 50 pcs upon request

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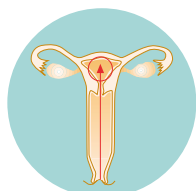
Cerviram

Endometrial sampler for cytologic screening

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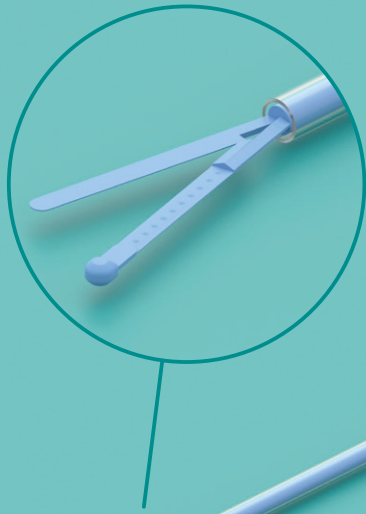 For a painless and safe endometrial cytologic sampling.

The protrusions on the asymmetrical forked tip allow an accurate and atraumatic sampling of the uterine tissue.

The outer sheath slides on the mandrel to facilitate insertion with closed tip. Avoids dilation and traction of the uterus neck.

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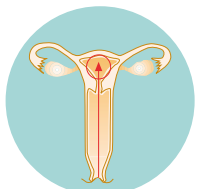
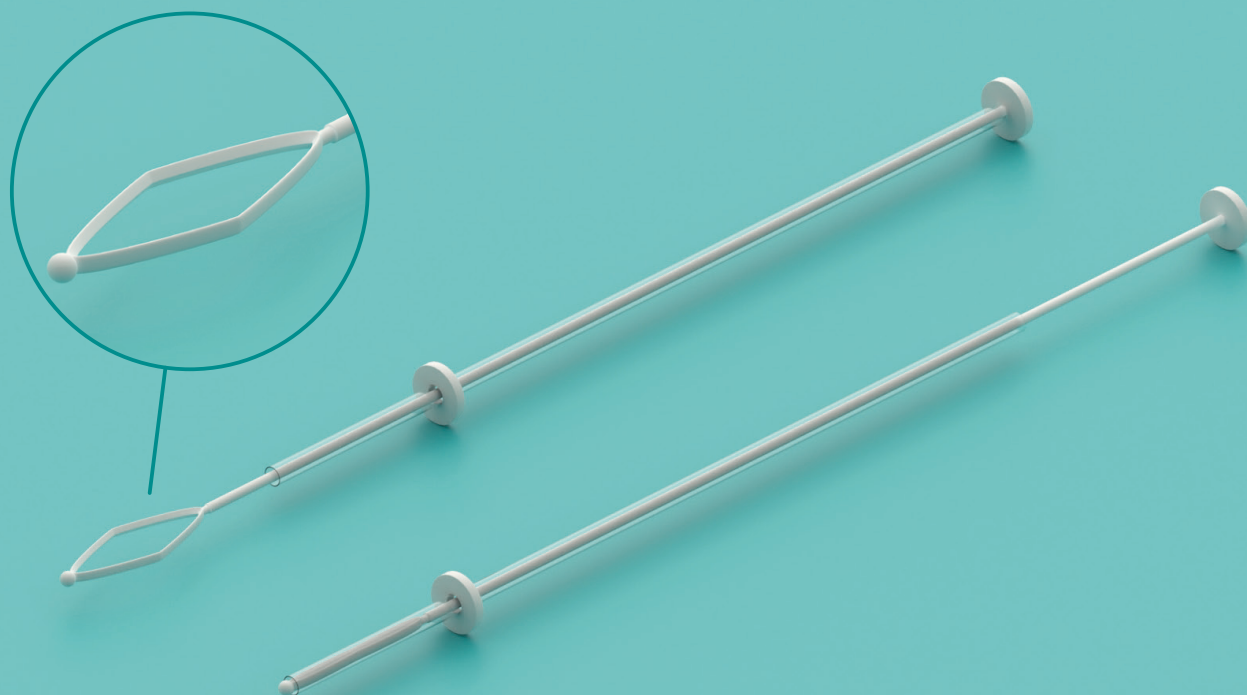
For a painless and safe endometrial bioptic sampling.

The rhomboidal shaped tip with sharp profile allows biopsy sampling of the uterine walls. The outer sheath slides on the mandrel to facilitate insertion with closed tip. An adjustable positioner is set on the outer sheath to determine the correct insertion depth. Avoids dilation and traction of the uterus neck.

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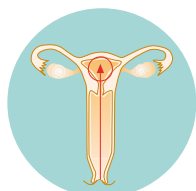
Ramcurette

Curette for aspirated endometrial samplin (Vabra method)

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For aspirated cyto-hystologic endometrial sampling and therapeutic curettage without anesthesia.

The kit includes:

- a curette with luer connector to be joined to a flow deflector provided with filter;
- a test tube to be connected to the deflector for sample collection.

The kit can be connected to an aspirator through a tube. The tip design allows for an accurate and effective procedure. Rinsing the curette tip inside the test tube at the end of the procedure enhances sample collection.

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Supplied in single packaging - disposable - EtO sterilization - CE 0051 marked

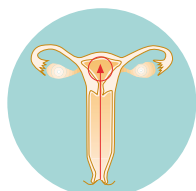
* Available in boxes of 25 pcs upon request

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
Endocic

*Endometrial sampling device (mucus - sperm)
with steel mandrel*

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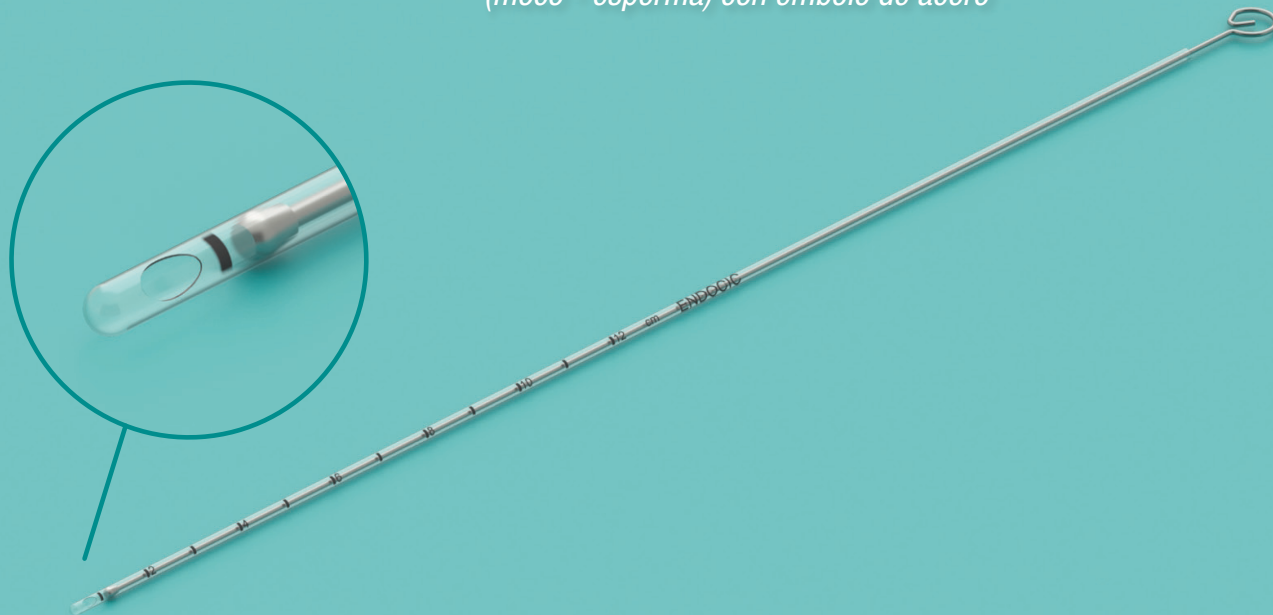 For a painless and easy endometrial cytologic sampling.

Sampling is performed manually by suction, through a mandrel that slides into an outer sheath. The steel mandrel allows for banding, enhancing suction intensity. Measurement scale ranging from 1 to 12 cm (1 cm resolution)*. The reduced external diameter makes it suitable for use in case of cervical stenosis. Avoids dilation and traction of the uterus neck.

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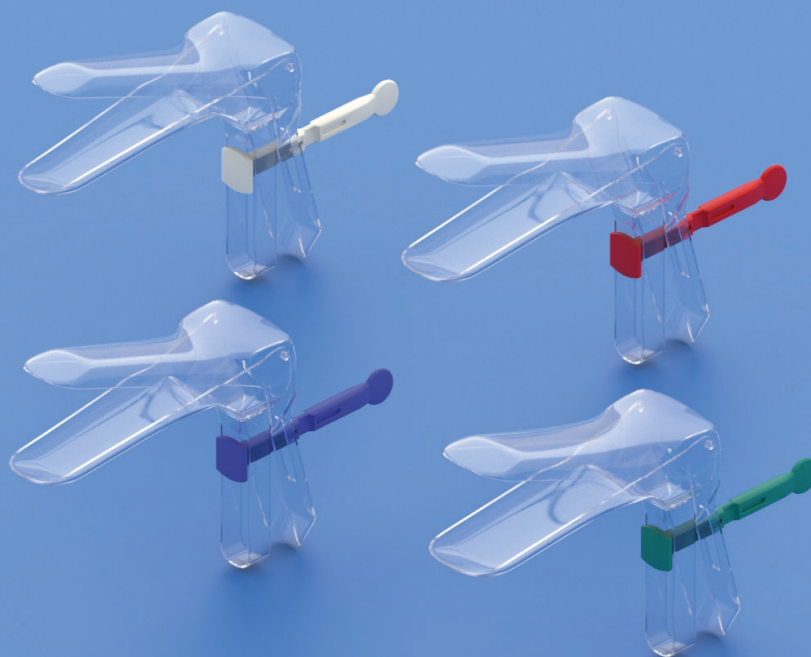
To widen the vaginal canal and expose the uterus neck.
The central key locking system allows a safe and fast locking of the valves.
The central key can be bent downward for an easier introduction of instruments.

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Supplied in single packaging - disposable - EtO sterilization - CE marked

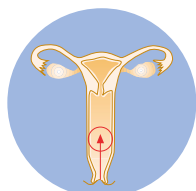
* Available in boxes of 60 pcs upon request

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Specuvag CS

Vaginal speculum with central screw-locking system (CUSCO model)

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Espéculo vaginal modelo CUSCO con bloqueo mediante tornillo central

 To widen the vaginal canal and expose the uterus neck. The central key locking system allows a safe and fast locking of the valves.

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
720106: disposable - EtO sterilization - CE 0051 marked
* Available in boxes of 50 and 100 pcs upon request (supplied in single packaging)
720214: disposable - non-sterile - CE marked

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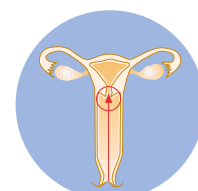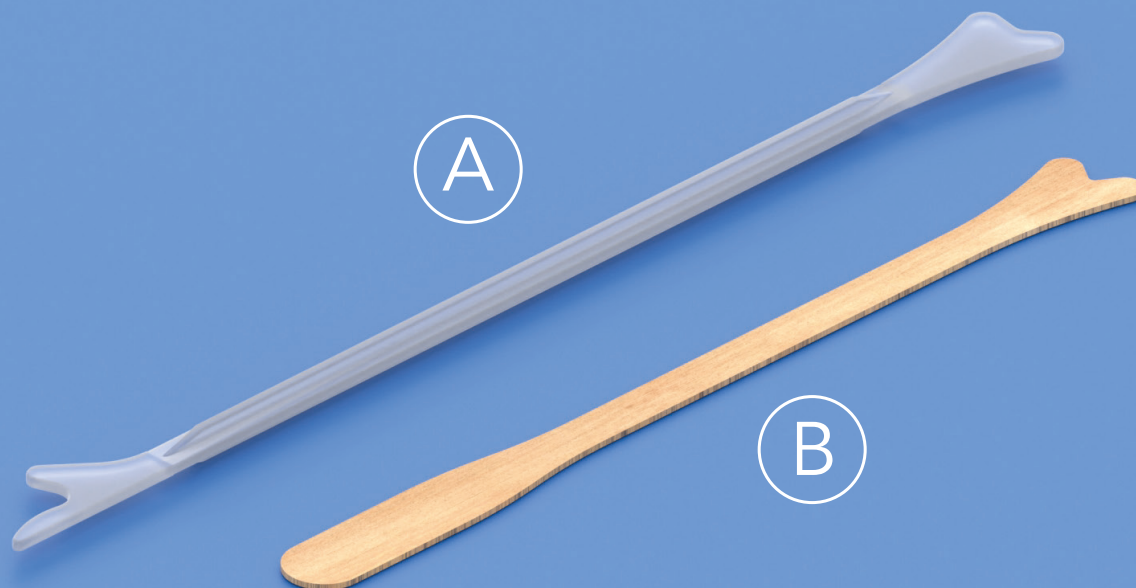
Wooden Ayre's spatula
Spatule Ayre en bois
Espátula de Ayre de madera

Ectocervix cells and posterior fornix sampler for cytologic smear.

The two differently shaped ends make the spatula suitable for specific purposes.

Both devices facilitate a 360° rotation and an optimal contact with the cervix.

Both Ramspatula and Ayre's spatula (in Birch wood) have smooth and splinter-free edges ensuring a completely comfortable procedure.

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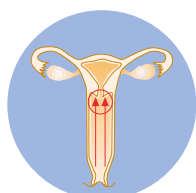
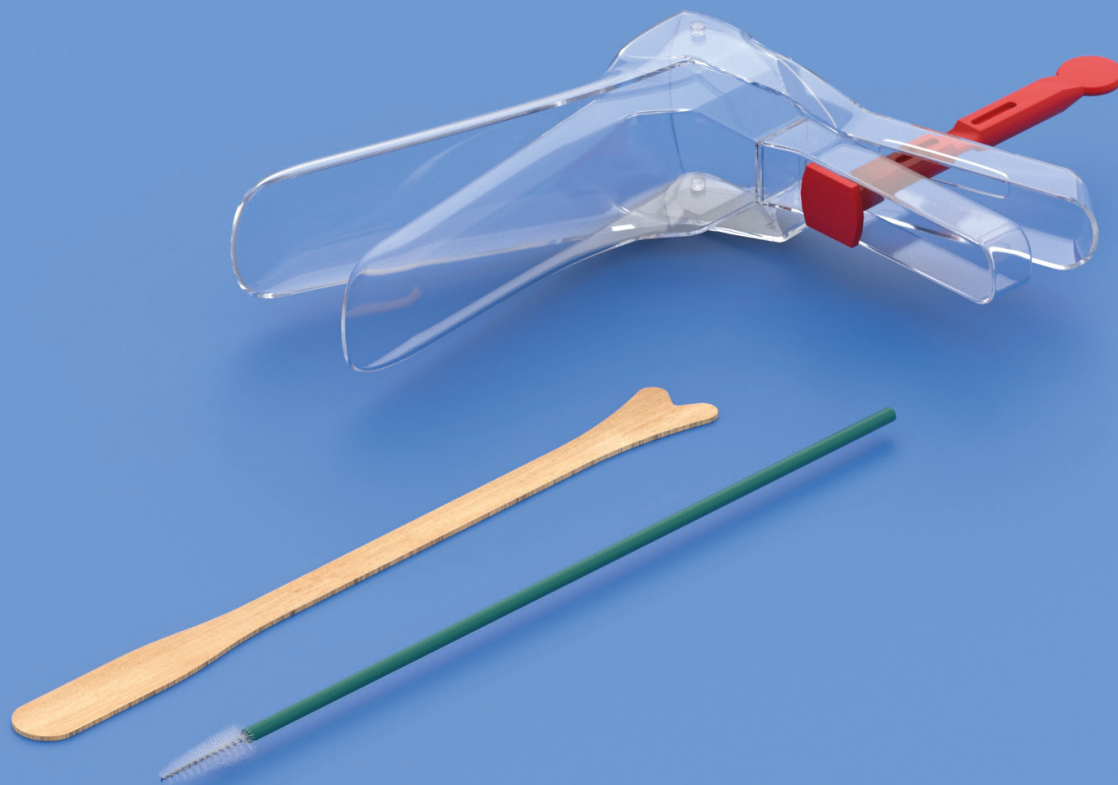
Pap-Smear is a painless screening test used to detect the presence of any abnormal cells on the cervix.

The functional configuration of the kit allows a ready-to-go operation. The standard kit includes:

- 1 SPECUVAG medium size speculum
- 1 wooden AYRE's spatula
- 1 RAMBRUSH for endocervical sampling

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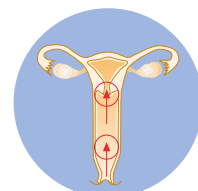
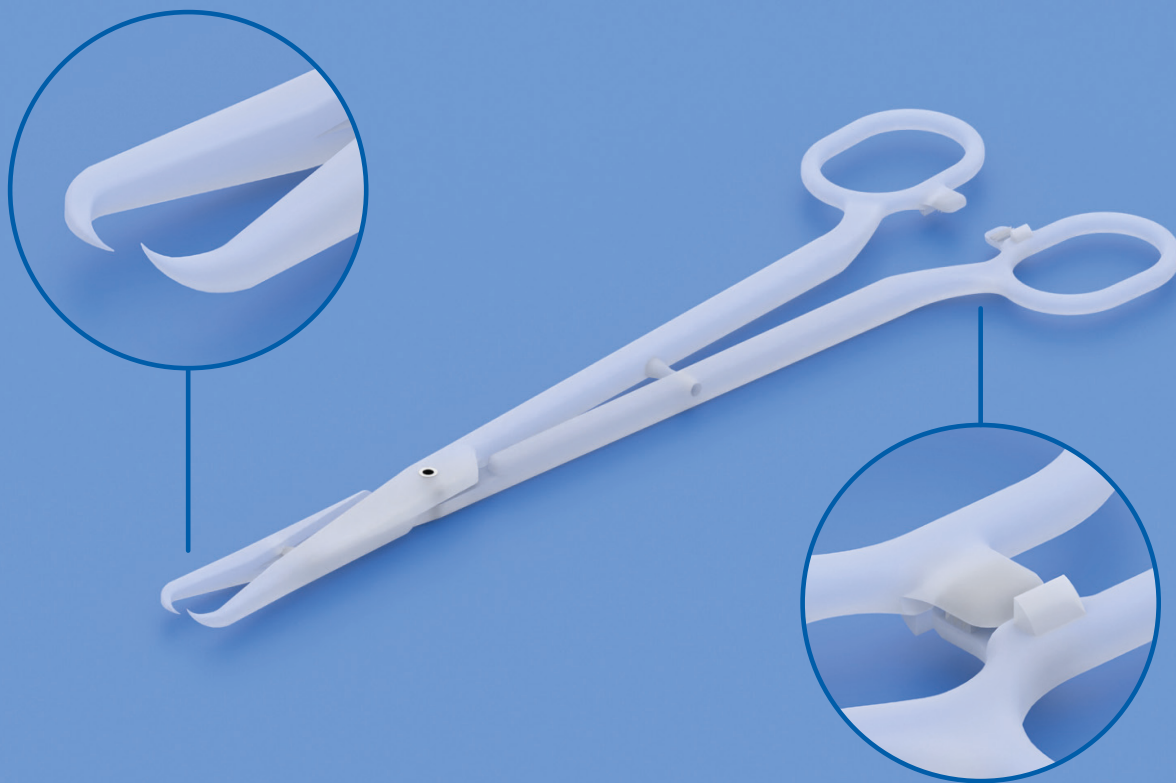
It is used in gynecological procedures where uterus neck grasping is required.

The pointed tips of the forceps and the triple locking clasp allow a safe and firm hold of the cervix.

The product design minimizes pain and bleeding during the procedure.

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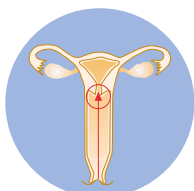
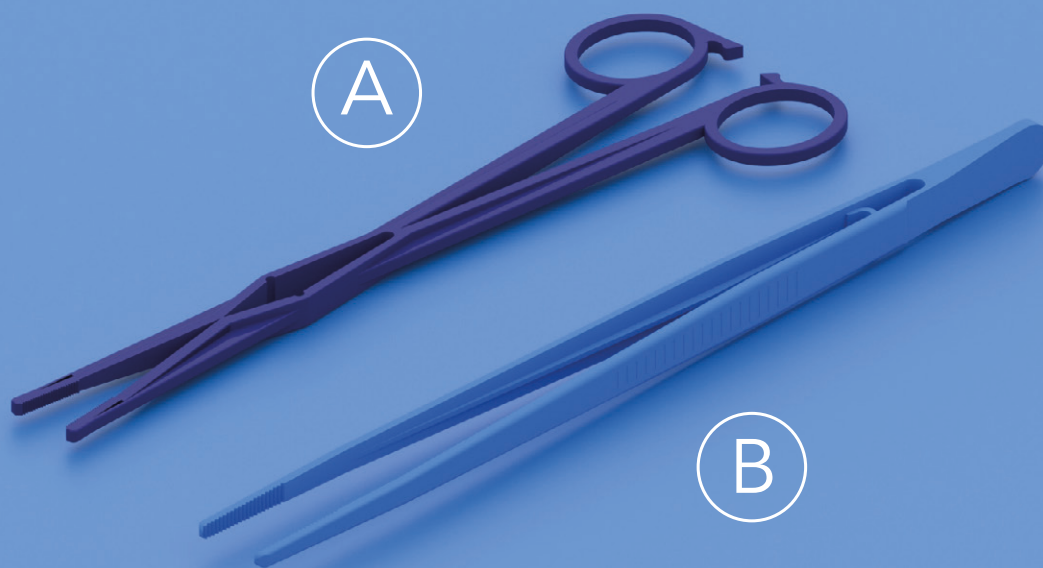
Pinzone

- **Kelly:** It is used for an easy handling of gauzes or dressings during medical procedures. It is provided with a self-locking system allowing hands-free operation. Thanks to its special clasp system it can be also used for the insertion and removal of intrauterine device.
- **Pinzone:** It is used to hold cotton flocks and gauzes during dressing procedures. Its elongated shape makes it suitable for gynecological procedures.

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
- **Kelly:** It is used for an easy handling of flocks or gauzes in dressing procedures. It is provided with a self-locking system allowing hands free. Thanks to its special clasp system it can be also used for the insertion and removal of intrauterine device.
- **Pinzone:** It is used to hold cotton flocks and gauzes in dressing procedures. It has a long shaped configuration particularly indicated for gynecological procedures.

Cod.A	Cod.B	Description - Descripción	Pcs/Box - Pièces/Boîte Uds./Caja
720174 sterile stérile estéril		Length 22 cm Longueur 22 cm Longitud 22 cm	50 pcs. (individually packed) 50 pièces (emballées individuellement) 50 uds. (envasado individualmente)
720974 non sterile non stérile no estéril			Bulk product in plastic bag of 100 pcs. Produit en vrac, sachets de 100 pièces Producto a granel en sobres de 100 uds.
	720183 sterile stérile estéril	Length 25 cm Longueur 25 cm Longitud 25 cm	100 pcs. (individually packed) 100 pièces (emballées individuellement) 100 uds. (envasado individualmente)
	720983 non sterile non stérile no estéril		Bulk product in plastic bag of 100 pcs. Produit en vrac, sachets de 100 pièces Producto a granel en sobres de 100 uds.

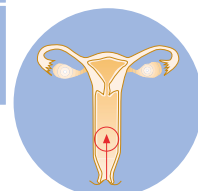720174 - 720183: disposable - EtO sterilization - CE 0051 marked
720974 - 720983: disposable - non sterile - CE 0051 marked
720174 - 720183: usage unique - stérilisation EtO - CE 0051 marked
720974 - 720983: non stérile - marqué CE 0051
720174 - 720183: desechable - esterilización por EtO - marcado CE 0051
720974 - 720983: desechable - no estéril - marcado CE 0051

Used to secure specimens for laboratory examination.

Frosted surface allows an easier data writing. Manufactured from white glass and pre-cleaned.

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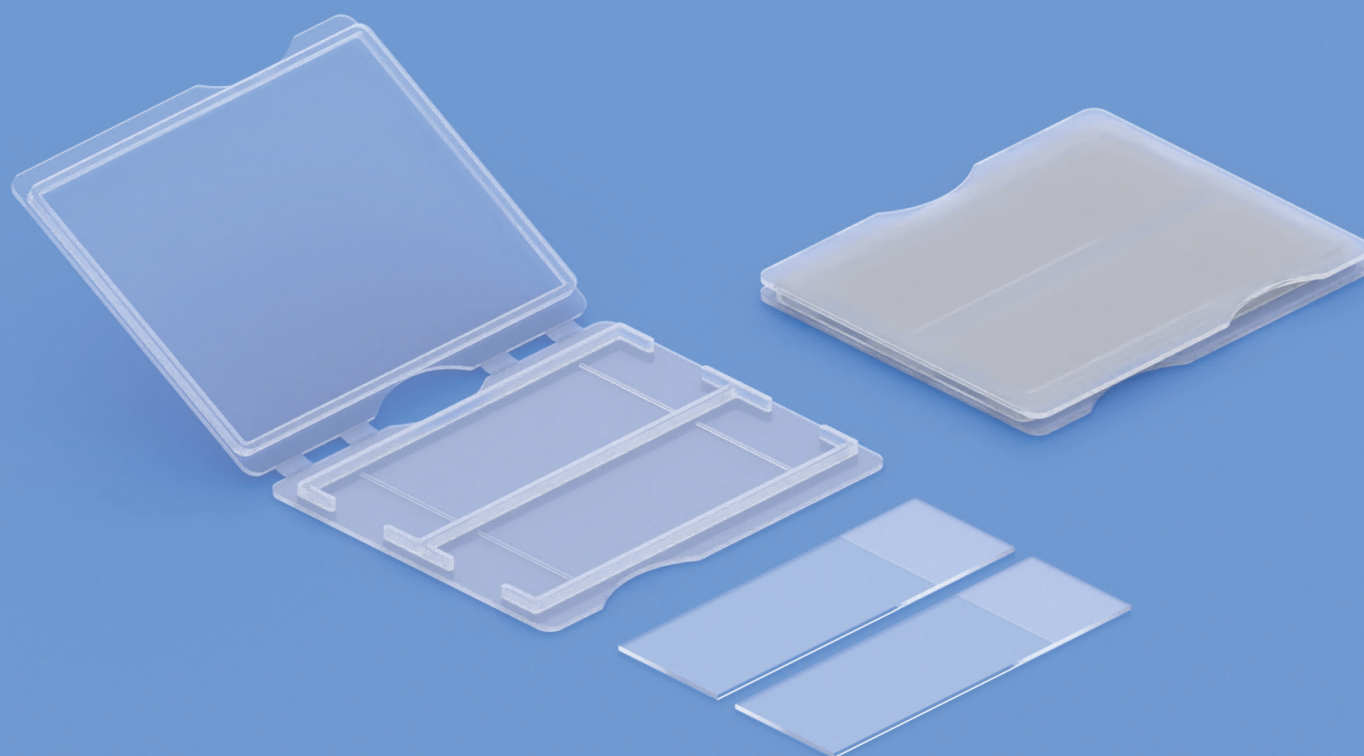
It stores up to 2 microscope slides with collected cells for cytologic examination.
It is manufactured in polypropylene and provided with a practical and safe locking system.

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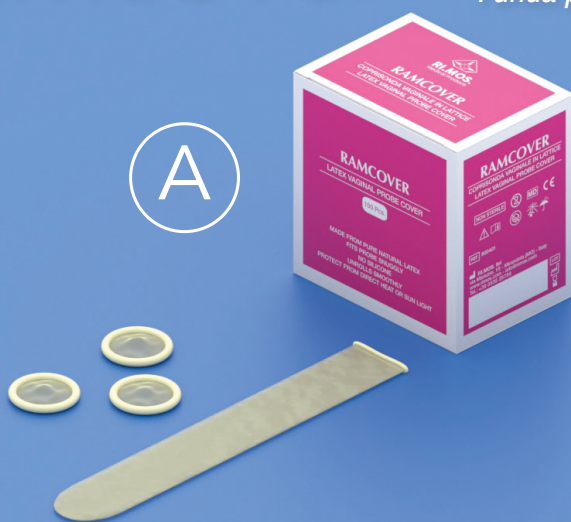
- **Ramcover:** To protect ultrasound transvaginal probe during ultrasonic examination. Made of natural latex. It is about 21 cm in length. Supplied non-lubricated and carefully rolled up with biocompatible powder for an easy unrolling onto the probe.
- **Stericover:** To protect ultrasound transvaginal probe during ultrasonic examination. Made of Eva (Ethylene Vinyl Acetate), latex free, non-cytotoxic. Supplied in à sterile single pouch.

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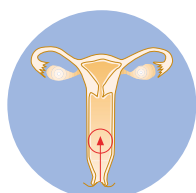
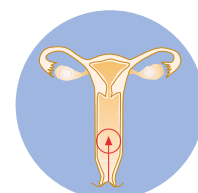
It allows an easy insertion of gynecological instruments.

The special composition and high transparency of the gel does not distort visual images when optical instruments are used.

It does not contain spermicides, it does not grease nor stain. It has an acid pH and a good electrical conductivity.

It is soluble in water. Thanks to its formulation, the gel is well absorbed by the skin and does not cause irritations to the mucous membranes.

It does not damage metals, plastic and rubber.

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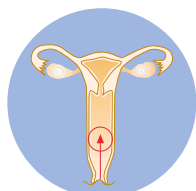
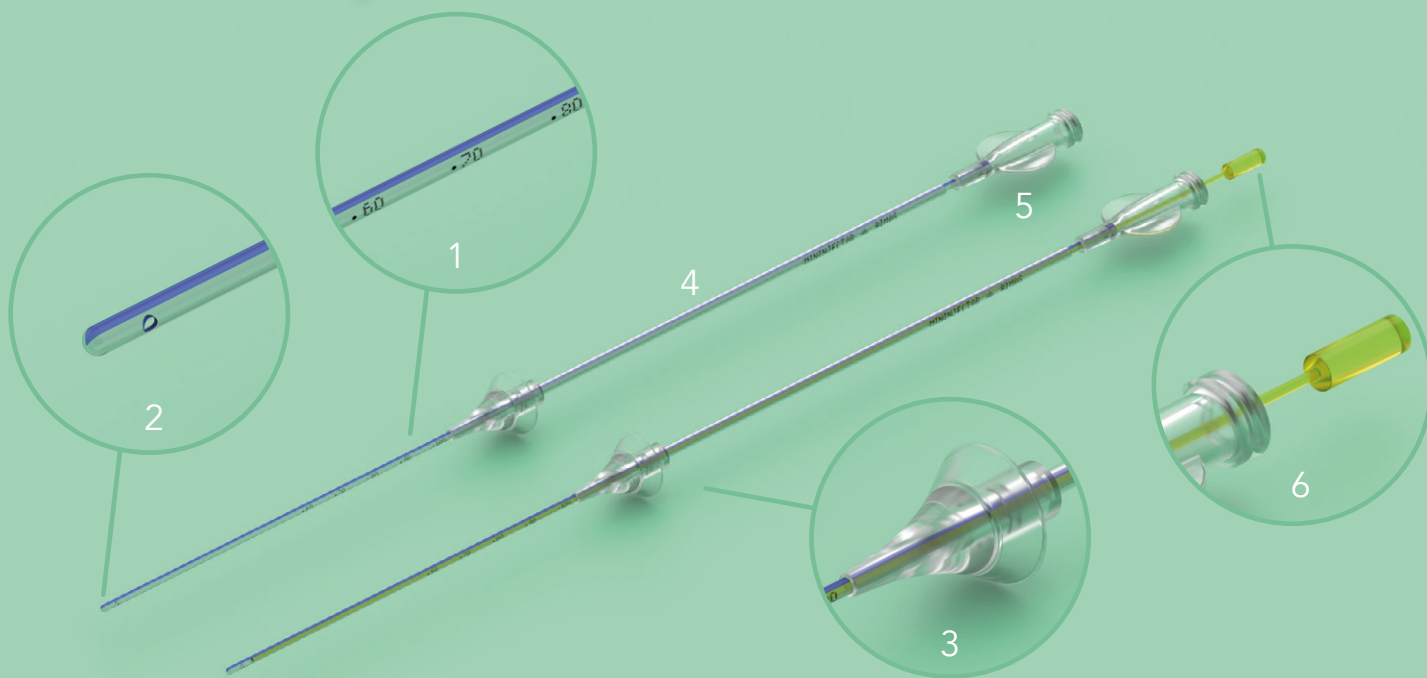
Mininjector CM

 To infuse saline solution in sonohysterography diagnostic procedures.

The device consists of:

- a clear catheter (1) with centimeter markings with blue radiopaque band for easy positioning;
- a side-holed soft tip (2);
- a soft rubber stopper (3) to ensure a perfect sealing on the uterus neck;
- a rigid introducer (4) to advance into uterus neck;
- a female Luer-Lock adapter (5) for syringe connection

Also available with mandrel (6) (**MININJECTOR CM**) for easier introduction into the uterine cavity.

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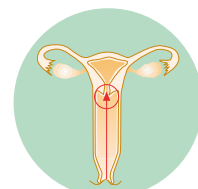
To inject contrast media (radio-opaque) in hysterosalpingography diagnostic procedures.

The catheters consist of:

- rigid introducer provided with a mobile positioner (2);
- soft double lumen radiopaque catheter (1) with a thermoplastic rubber oval balloon of 2.5 ml on the distal tip, that ensures a firm positioning of the catheter;
- stopcock (4) to regulate balloon inflation;
- tubing clamp to avoid liquid backflow (5);
- female Luer-Lock connector (6) for contrast media injection;
- a 2.5 ml syringe (3).

Available in 2 versions:

SOFTINJECTOR (A) with soft cathete, **RIGIDINJECTOR (B)** with rigid catheter.

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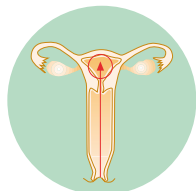
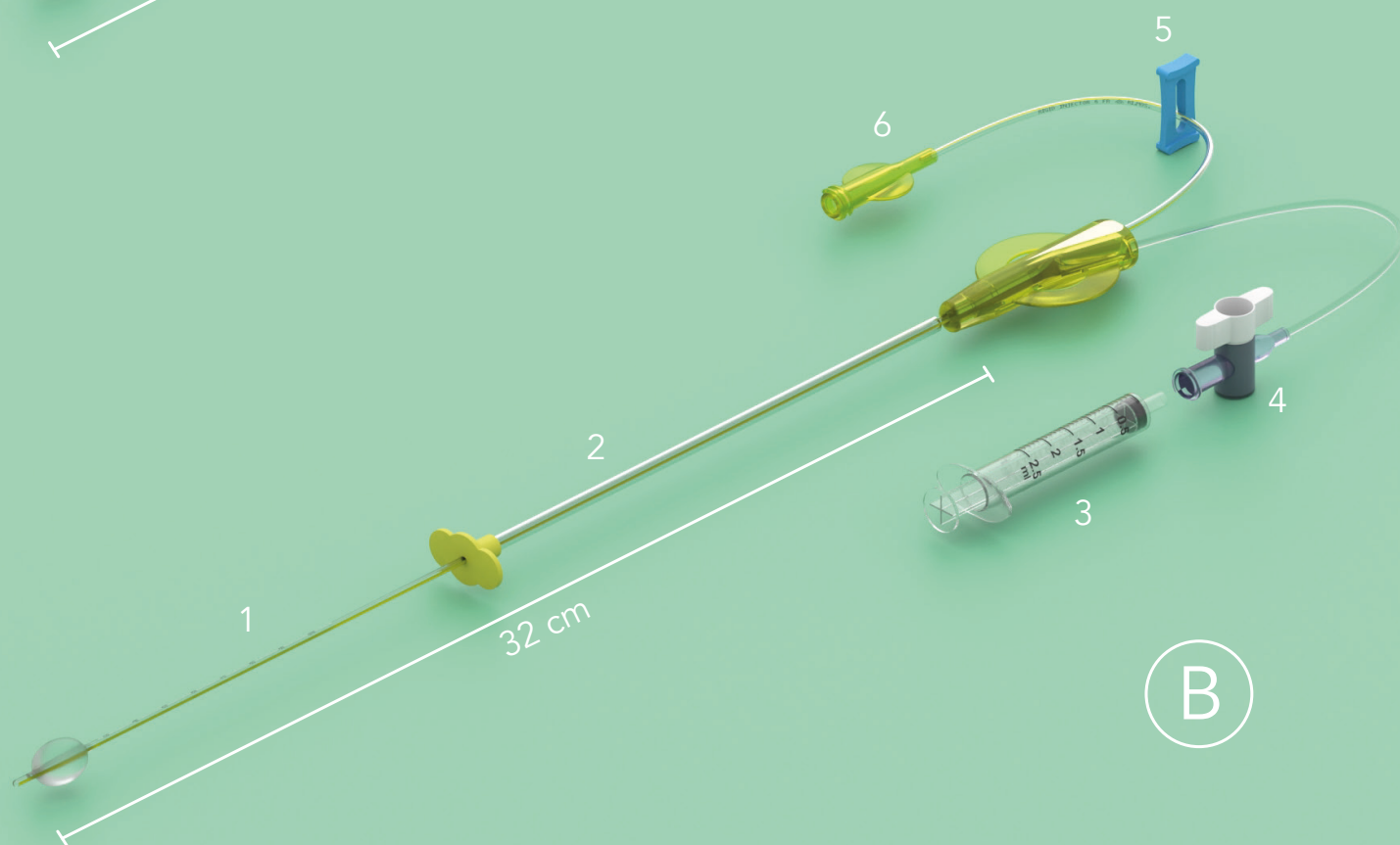
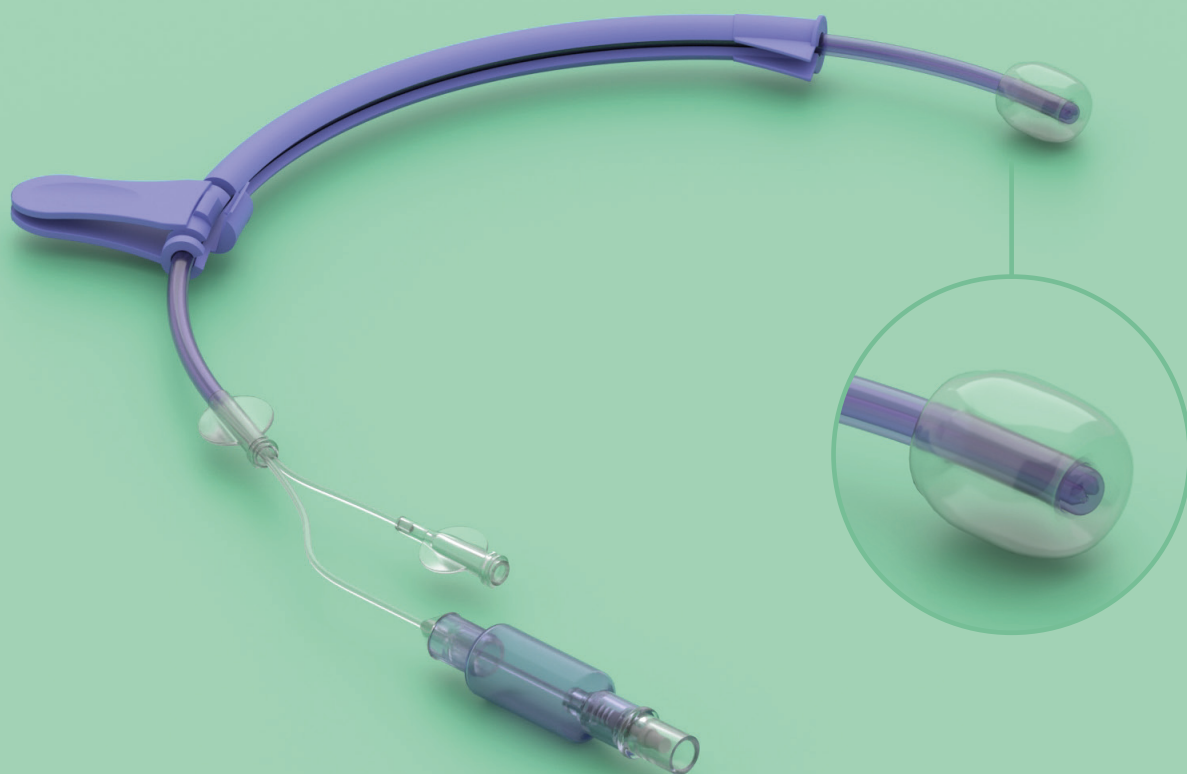
It is used for procedures where uterus manipulation and/or injection of contrast media is requested (e.g. diagnostic laparoscopy, fertility examinations, etc.). It features a curved shape for an easy insertion. It is composed of:

- double lumen inner catheter with graduation from 4 to 10 cm (1 cm resolution)* to ensure correct insertion depth;
- stiff handle allowing a firm and safe manipulation;
- simple locking system to avoid accidental movements during the procedure;
- 10 cc PVC balloon with diameter 20 mm placed on the catheter tip to minimize the risk of accidental perforation and prevent contrast media reflow;
- one-way inflation valve directly connected to a pilot balloon showing the real-time condition of the inner balloon;
- luer lock connector for contrast media injection.

* It gives an approximate measurement.

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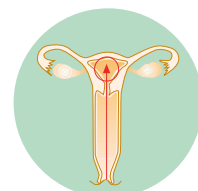
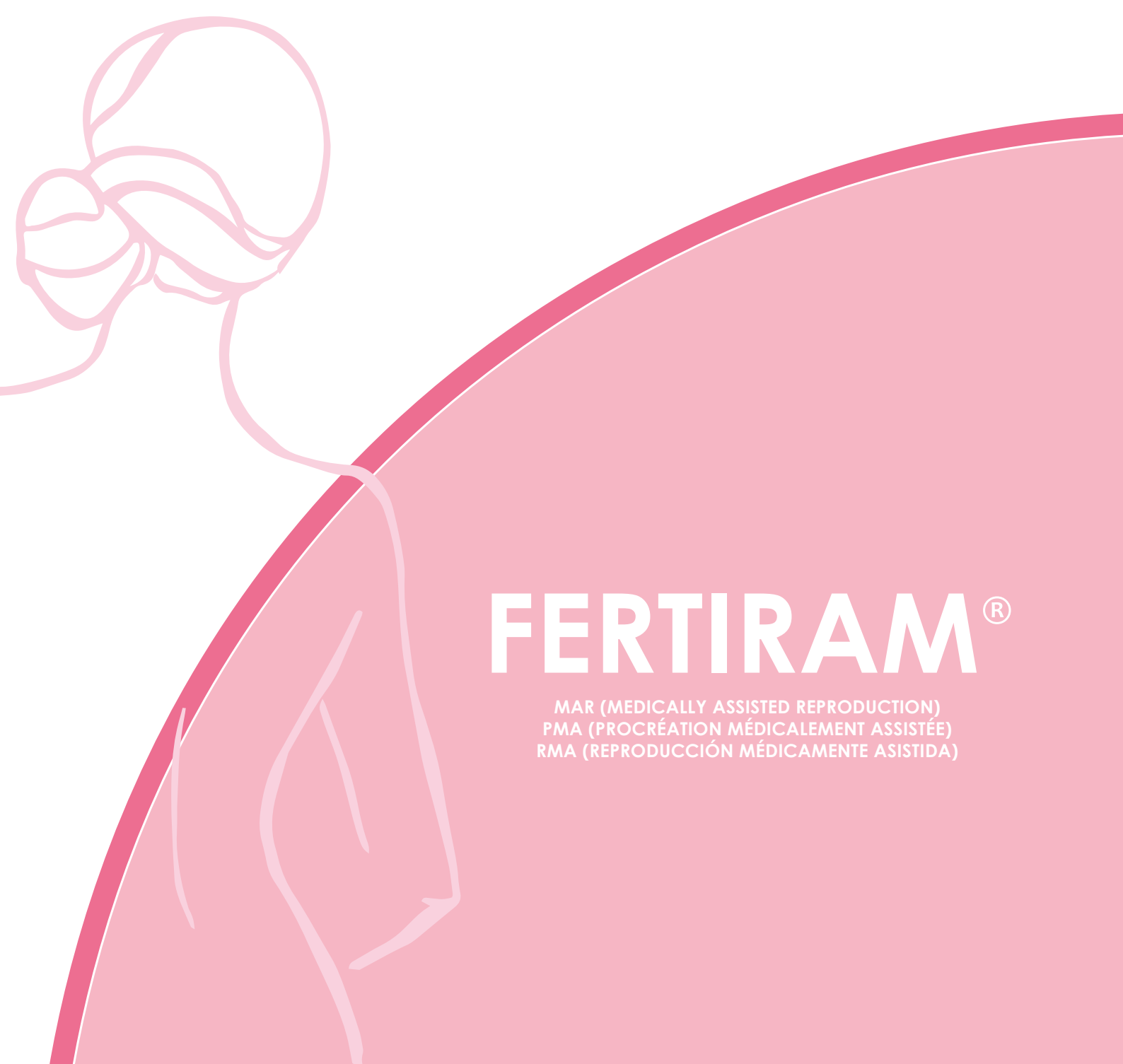
Ainsewhite

For smooth transfer of washed sperm into the uterine cavity, during Artificial Insemination procedures.

Centimeter-marked* catheters (3) with rounded tip (4) and double side-eyes. A Luer-Lock adapter serves as finger grip and allows a safe syringe connection. The two orientation wings (2) aligned with the double side-eyes indicate orientation during the placement.

Both catheters are not intended for in vitro fertilization and Intra Fallopian tubes procedures. Available in 2 versions:

- **AINSEGREY (A)** IUI centimeter-marked* soft catheter with steel mandrel (1) to facilitate insertion.
- **AINSEWHITE (B)** IUI centimeter-marked* soft catheter.

* It gives an approximate measurement.

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The catheter features a single lumen, eco-guided transparent design with differentiated degrees of stiffness:

- rigid proximal portion (2) for easy insertion through the cervix;
- flexible, soft, distal tip (3), for atraumatic and gentle seminal transfer.

Centimeter-marking catheter (50-60-70 mm)*. A Luer-Lock adapter (1) serves as finger grip and allows a safe syringe connection.

The two orientation wings aligned with the double side-eyes placed on the distal tip indicate orientation during the placement.

Available in 2 versions:

- **AINSEBLUE (A)** double side-eyes allow an excellent distribution of the seminal fluid.
- **AINSEBLUE-T (B)** seminal fluid injection occurs through an open tip.

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
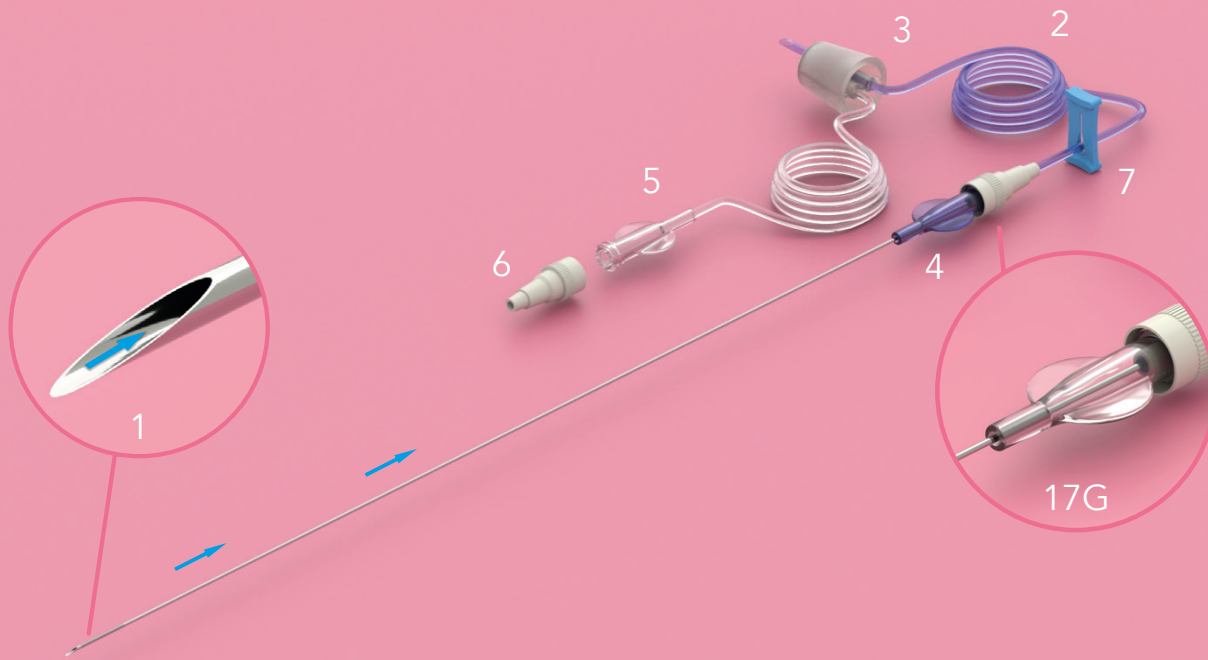
Single lumen, one-way needle composed of:

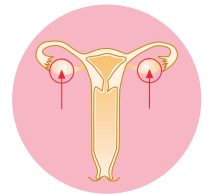- echogenic tip with atraumatic bevel (1);
- blue-colored (2) standard suction tubing 100 cm long;
- thermoplastic medical rubber bung (3) connected to the test tube;
- two orientation wings (4) aligned with tip bevel to orient the operator during penetration phase.
- oocyte aspiration is regulated by a tubing clamp (7).
- 80 cm pump connection tubing ending in a Luer-Lock adapter (5).
- an additional universal adapter (6) is included to allow multiple connections.

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Doublecyte-N

Double lumen needle for oocyte aspiration and follicle flushing

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Double lumen, double-way needle, one-way for oocyte aspiration (6), second way for follicle flushing (4) consisting of:

- echogenic tip with atraumatic bevel (1);
- oocyte aspiration is performed through the inner lumen of the needle (2) and extends through a blue-colored (6) standard suction tubing 100 cm long;
- a thermoplastic medical rubber bung (7) connected to the test tube;
- a Y connector (3), serving as finger grip, provided with two universal adapters connecting the needle to aspiration and washing tubing set;
- follicle flushing is performed through a plastic tubing set 100 cm long (4) ending in a stopcock (5) (washing flows through the outer lumen of the needle);
- oocyte aspiration is regulated by a tubing clamp (8).
- 80 cm extension ending with a Luer Lock adapter for connection to the suction pump (9).
- un adaptateur universel supplémentaire est inclus (10) pour une variété de connexions.

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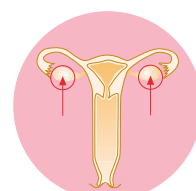
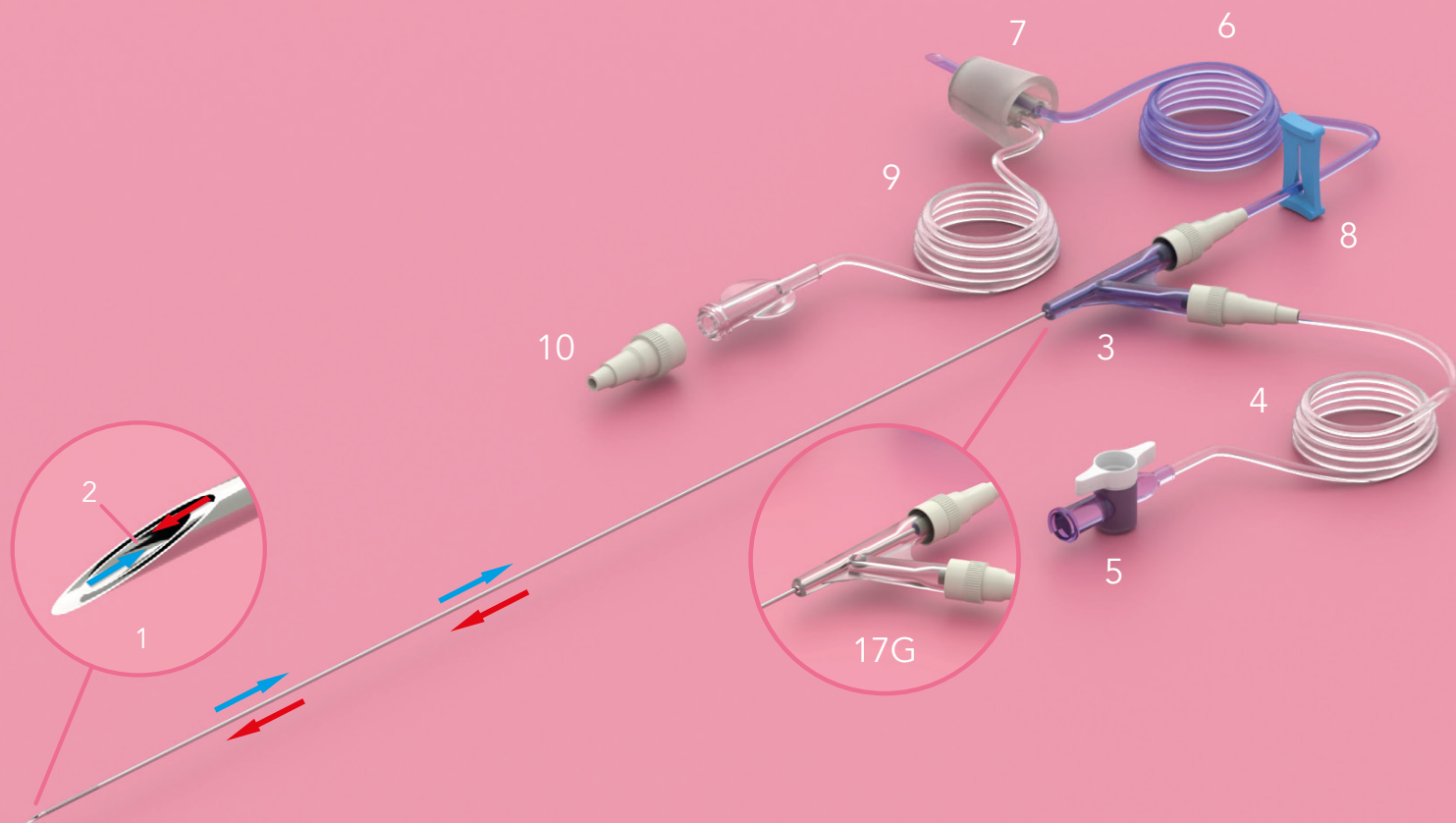
- 

Recommended for cases of cervical stenosis thanks to its flexible stainless-steel mandrel.
The embryo transfer set is composed of:

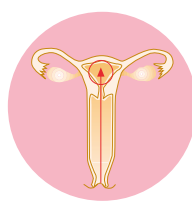
 - a light blue colored outer introducer (3) with graduated markings placed at 5-6-7-8 cm from the tip;
 - a soft and clear distal tip (4) is tapered (8) for ease of insertion.
 - a malleable mandrel allowing catheter shaping once it is inserted (1);
 - a Luer-Lock connector that serves as finger grip (2);
 - an embryo transfer catheter consisting of a rigid proximal portion (5), a soft distal portion (6) with a tapered tip facilitating an eco-guided insertion (7). Available in 2 different sizes: long **TRANSFER OMNI-L (A)** and short **TRANSFER OMNI-S (B)**.
- 

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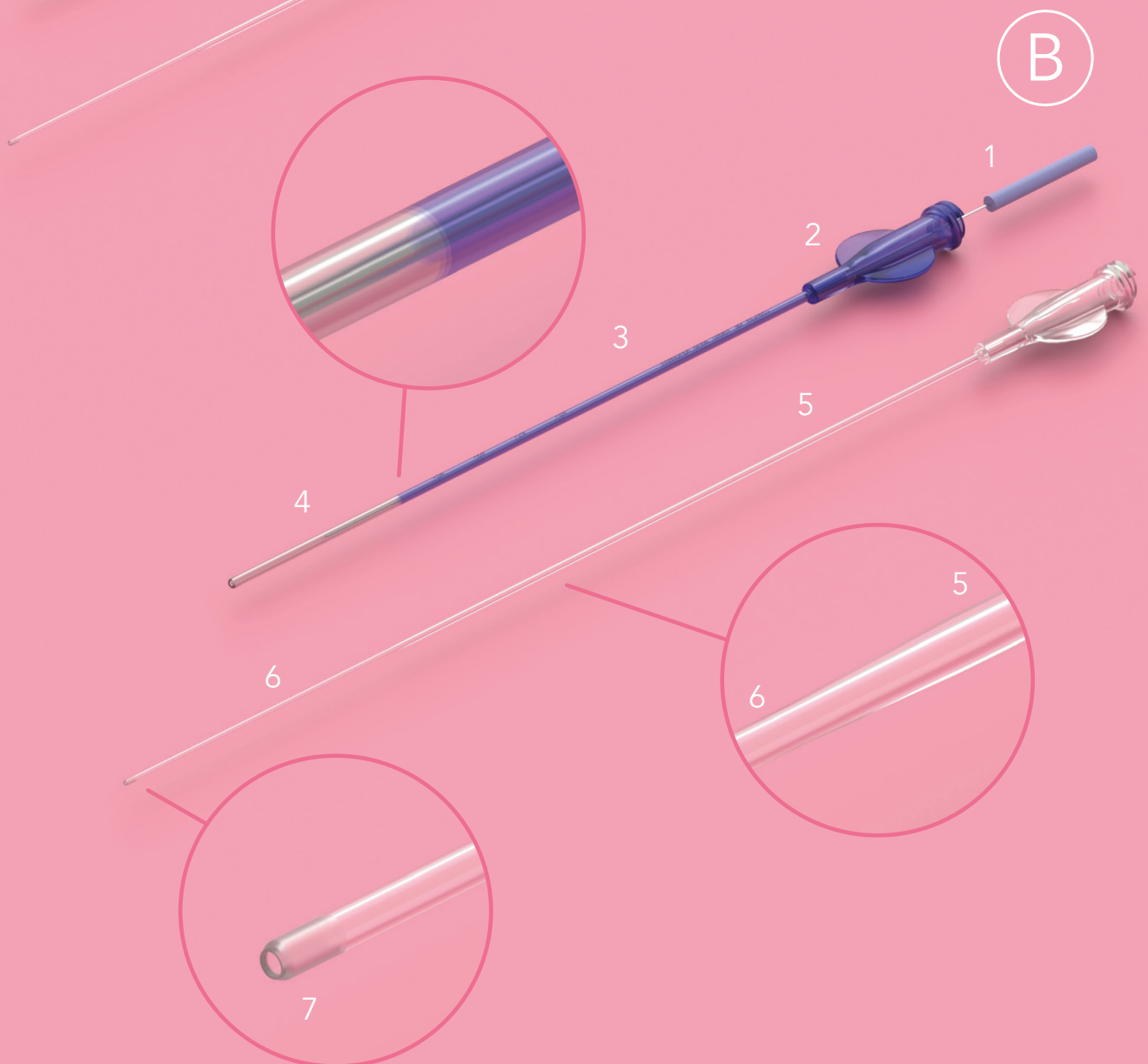
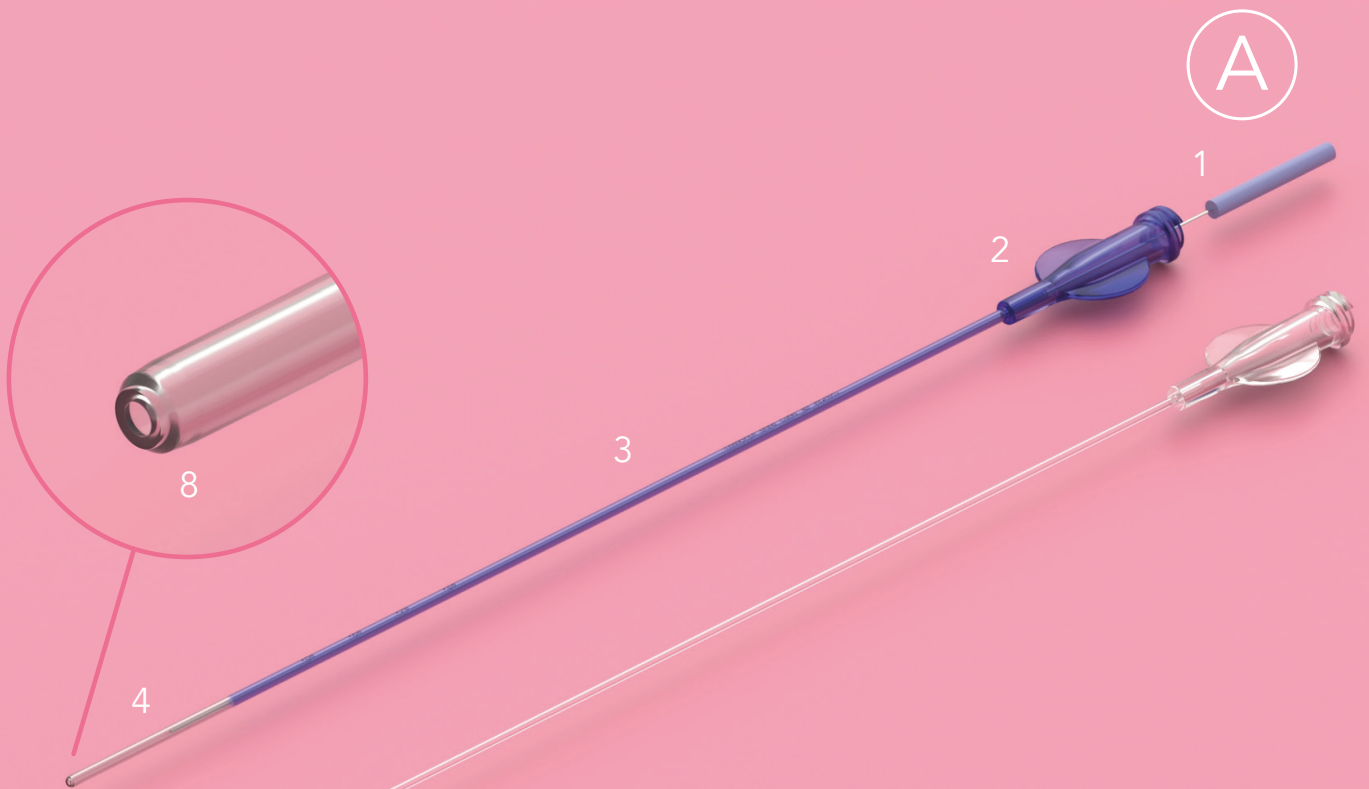
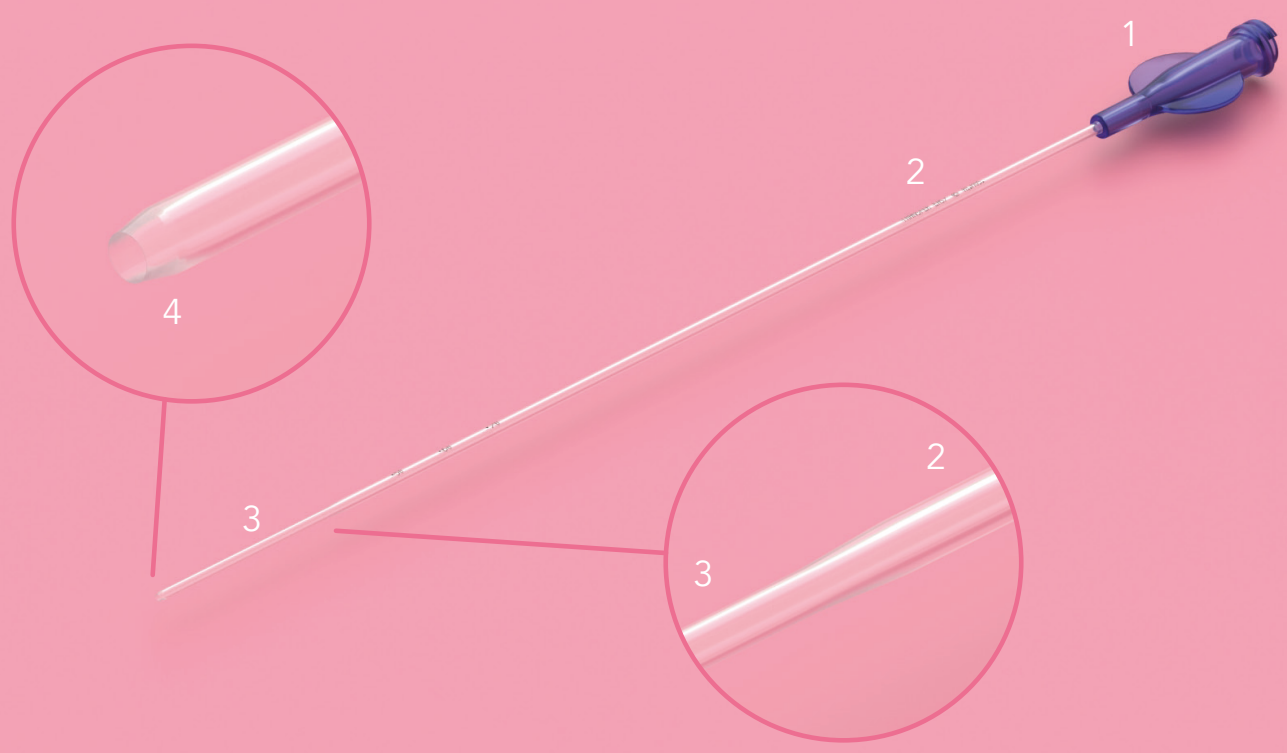
Ideal for cases where the permeability of the cervical canal is considered normal.
The catheter consists of:
- a rigid proximal portion (2);
- a soft and flexible distal portion (3), with echogenic tip (4) facilitating an eco-guided insertion.
- markings at 5, 6 and 7 cm from distal portion aid in correct placement into the uterine cavity.
- a Luer-Lock connector (1) serves as finger grip and allows a safe syringe connection.

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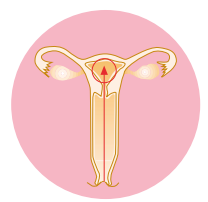
BUSINESS POLICY

INTERNATIONAL CUSTOMERS

If you are a final user (doctor, nurse, hospital, etc.). Contact us at info@rimos.com and let us know your country: we will inform you about the nearest dealer in your country and you will be contacted directly. In some selected cases where a distributor is not present in the country, RI.MOS. might decide to supply the end-user directly.

If you are dealer, importer or distributor. Our business policy is to sell our products through distributors: we usually do not sell directly to public and / or private hospitals and clinics. International sales are currently managed worldwide through a network of distributors: if you want to join our sales network you are welcome. Our favorite dealers / distributor profile is the following:

- Competence and experience in marketing and sales of medical products;
- Expertise and introduction in the local market;
- Capability to outline and develop a marketing action plan and / or working program in order to become a reference point for the final users.

Contact us at info@rimos.com for more information and let us have your company profile.

Samples Policy. Due to systematic samples request from any inquiring company, RI.MOS. cannot afford to give free samples unconditionally. Therefore the following samples policy is applied:

- Products priced under 5 Euro: 1 free sample is given; additional samples (up to 10 pieces per type) are priced at 50% of current list price; products priced over 5 Euro: 1 sample is given with a 50% discount on current list price; additional samples are fully priced;
- The buyer is usually charged with relevant shipping costs. Freight collect shipments are preferable; please remember to advise the name of your courier service with your account number.
- Samples payment can be done as follows:
 - in advance by bank transfer (see bank details on the right)
 - credit card (only VISA or MASTERCARD)
 - Paypal

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GENERAL CONDITIONS OF SALES

- 1. Products.** All dimensions and technical features mentioned in RI.MOS. catalogue is pure reference information. All Products listed in this catalogue are CE marked. All materials used for disposables satisfy current international standards for plastic materials used in medical devices.
- 2. Price list.** Products prices are indicated in RI.MOS. Export Price List. RI.MOS. reserves the right to apply a 10 EUR surcharge in presence of purchase orders under 200 EUR.
- 3. Currency.** All the prices are expressed in Euro (EUR).
- 4. V.A.T., taxes and duties.** Selling prices do not include V.A.T., state, federal and governmental taxes and duties. The BUYER will be liable for such taxes if applicable.
- 5. Delivery charges.** Selling prices, unless otherwise agreed between the parties, do not include freight charges (prices are to be intended EX WORKS MIRANDOLA - INCOTERMS 2010). The BUYER shall specify requested delivery term: (EX-WORKS, CPT, CIP, CFR, CIF, DAP). In case of ex-works delivery, the BUYER shall advise name of his favourite forwarder while placing the order. In all other cases, Rimos will choose the cheapest forwarder and relevant freight costs will be invoiced together with the products.
- 6. Validity of the prices.** All the prices are valid for 60 days from the date of the offer, unless otherwise agreed.
- 7. Purchase orders.** The BUYER shall submit purchase orders by e-mail or fax and mention:
 - code no., description, quantity (per product code);
 - requested delivery date; shipping and packaging instructions; shipping address.
- 8. Acceptance and confirmation of orders.** RI.MOS. shall send a proforma invoice as order acknowledgement to the buyer upon receipt of the purchase order. The buyer is requested to sign and return the proforma invoice as order confirmation to RI.MOS.; the order will be considered null and void when no proforma invoice is returned or no prepayment is received.
- 9. Delivery terms.** Shipment of the Products shall be arranged within seven (7) days from order receipt when the ordered products are available in stock. Out-of stock items are shipped within thirty (30) days from order receipt, unless otherwise agreed between the parties. Delivery terms may be longer when prepayment is requested.
- 10. Modification of orders.** Confirmed orders cannot be modified or cancelled by the buyer except upon written agreement of both parties.
- 11. Minimum orders required.** The minimum purchase quantity is one unit case (and / or relevant multiples). In presence of small orders under 200 EUR a 10 EUR surcharge will be included.
- 12. Complete order.** Purchase orders are considered shipped complete when ordered quantity and invoiced quantity coincide.
- 13. Payment Terms.** Prepayment is required with the order unless otherwise agreed between the parties.
- 14. Payment Method.** Payment can be made by: SWIFT wire transfer, letter of credit irrevocable confirmed by first class bank (only in presence of considerable amounts), and credit card (only VISA or MASTERCARD). Payment by credit card involves a 3,5% commission surcharge. All information about RI.MOS. Bank and Account number shall be detailed on order acknowledgement.
- 15. Warranty.** RI.MOS. warrants Products shall be free from defects in materials and workmanship for a period of 12 (twelve) months from shipment date. RI.MOS. disclaims any responsibility for damage or injury caused by the Products if used by unauthorized personnel. Any unauthorized use during the warranty period will immediately make the warranty void. If the Products are defective because of an original defect in materials or workmanship, they will be replaced by RI.MOS. free of charge. The Products that are defective due to negligence or improper use, incorrect use, careless transportation and handling, changes in the materials or in the dimensions of the Products caused by the exposure to chemicals, and / or inappropriate temperatures during storage of the Products without taking the necessary precautions, or to any circumstances that are not the result of manufacturing defects, are not covered by this warranty. Should the Products have manufacturing defects, the warranty covers the replacement only. However, replacement and compensation for direct or indirect damage or damage of whatever nature to persons or things, are not covered by this warranty. RI.MOS. is not liable for possible damage or injury that could result, directly or indirectly, to persons or things as a result of lack of observance of the provisions indicated in the instructions for use and especially concerning Product's use, shipping and storage instructions. Any defective Products replaced under this warranty will remain property of RI.MOS.
- 16. Return of goods policy.** Any claim about forwarding, delivery, quantity and quality of the goods has to be advised within eight (8) days from the delivery date of the goods. RI.MOS. accepts any claim for shortage or damaged goods only if presented together with the copy of the delivery note, showing the reservations expressed to the carrier upon goods receipt. Any return of goods must be previously authorized by RI.MOS. in accordance with its internal procedures. Goods must be returned in the original packaging and cartons have to be properly sealed. All shipping charges related to returned goods from BUYER to RI.MOS. are paid by the BUYER, unless otherwise agreed between the parties.
- 17. Force Majeure.** Neither party shall be liable to the other party for any loss, injury, delay, damage or other casualties suffered or incurred by such other party due to strikes, riots, storms, fires, acts of God, war or any other cause beyond the reasonable control of either party.
- 18. Disputes.** Any dispute, controversy or claim arising out of or in connection with the present general condition of sale which is not settled friendly, shall be finally settled in accordance with "il Codice di Diritto Italiano" (Italian law); place of jurisdiction is Court of Modena (Italy).

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