



RI.MOS. | VET

Hypermix[®]



**CATALOGUE RESERVED EXCLUSIVELY FOR VETERINARY
DOCTORS AND PHARMACISTS**

www.hypermix.it



FOR BROKEN SKIN. MOISTURIZISE AND SOOTHES.
IT IS UNPLEASANT TO INSECTS, therefore it finds wide field of application in animals living outdoors.

HYPERMIX® is a topical treatment that promotes the regeneration of external wounds.

ACUTE - CHRONIC - INFECTED



PROCESSING AIDS
IN THE REPAIR AND
REGENERATION
OF THE INJURED SKIN



CONTROL
OF THE BACTERIAL LOAD



PROTECTION OF THE INJURED
OR REDDENED SKIN

The combined effect of the two plants contained in **HYPERMIX®** is greater than the sum of the their individual effects (bibliography available on request).



NEEM (Azadirachtin)



Antiseptic
Antimicrobial
Antibacterial
Antifungal



HYPERICUM (Hyperforin)

Regenerating
Re-epithelializing
Soothing




CLINICAL CASES - PET

Day 0



Day 25

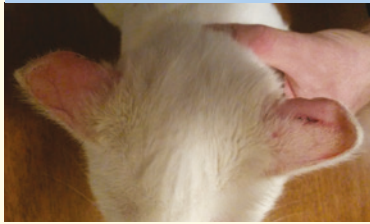

DERMATITIS - Course: 25 days

CAT - Photosensitization dermatitis on both ear pavilions. Treatment with **Hypermix® GEL** 2 times a day, preceded by cleaning with saline solution.

Courtesy of: **Dr. Marco Lapia - ENPA of Trieste**

Before curettage



Day 28



WOUND - Course: 28 days

DOG - Deep wound probably from leghold trap.


After cleaning and bath in warm water to reactivate the circulation and after surgical curettage, therapy began with daily laser with application of **Hypermix® GEL** and bandage.

Courtesy of: **Amb. Vet. Ass. SANTO STEFANO - Dr. Biondi S. and Dr. Mercuri M.**

Day 0



Day 35

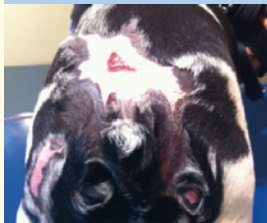

BURN - Course: 35 days

FRENCH BULLDOG - After curettage of the area,

it was prescribed an antibiotic therapy with clindamycin, bi-daily washes with mild soap and application of **Hypermix® OIL** formulation.

Courtesy of: **Dr. Antonio Silvestri**

CLINICAL CASES - EQUINES

Day 0



Day 21



ONYCHOMYCOSIS - Course: 21 days

THOROUGHBRED - Compromised hoof with wall detachment from the underlying tissues.

After cleaning of the sole from the worm, treatment with **Hypermix® OIL** at least once a day.

Courtesy of: **Rosario Guida, Equisud**

Day 0



Day 60




THIGH INJURY - Course: 60 days

THOROUGHBRED BREEDER - Injury to the forearm.


Application of **Hypermix® OIL** once a day.

Courtesy of: **Dr. Riccardo Rossi**

Day 0

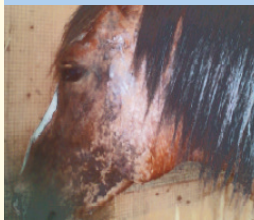

Day 30

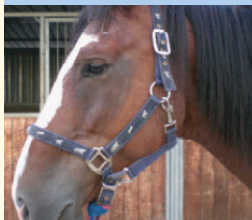

DERMATITIS - Course: 30 days

HORSE - Summer dermatitis.

The first seven days were sponged with **Hypermix** without washing or brushing the coat.

In the following days, the horse was washed and dried well in the affected part and it was applied **Hypermix**.

After each application the horse was exposed to the sun (not during peak hours). In total 150 ml of **Hypermix®** was used.

Courtesy of: **Rosario Guida, teacher of Mascalucia 4 level**

CLINICAL CASES - EXOTIC

Day 0




Day 40



WOUND - Course: 40 days

DWARF RABBIT - Traumatic bite wound in the lateral cervical area. Immediate therapy: irrigation with warm physiological solution. Manual removal of necrotic tissue fragments. Application of **Hypermix® OIL** twice a day, daily application. Courtesy: **Dr. Gianluca Marchetti**

Day 0

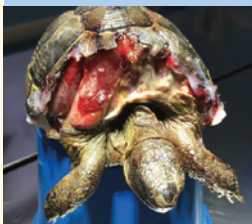

Day 60




INJURIES - Course: 60 days

TESTUDO HERMANNI (Female -adult) - Severe bite injuries. In the first 10 days intramuscular antibiotic therapy, anti-inflammatory, painkiller and intracelomatic rehydration. After cleaning was applied an antibiotic ointment and **Hypermix® GAUZE** (once a day for 7 days). With the appearance of granulation tissue, local dressing only with **Hypermix® GAUZE**: at the beginning daily application, then every other day for about one month. With the formation of scar tissue, we switched to weekly application of **Hypermix® SPRAY**. Courtesy of: **Dr Diana Cassano DVM GPCert (ExAP)**

Day 0



Day 6

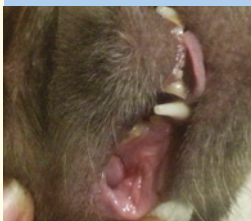

GINGIVITIS - Course: 6 days

SKUNK - Bacterial gingivitis. Application of **Hypermix GEL** twice a day. Courtesy of: **Dr. Riccardo Mancini**
Circus activities "Zoo Safari, La Favola Siamo Noi"

Day 0



Day 25



DERMATITIS - Course: 25 days

Python molurus bivittatus - Positive to pseudomonas aeruginosa. Treated with Marbofloxacin for 10 days, disinfected every 48-72 hours with chlorhexidine gel, curettage of necrotic tissues, until moulting. Application of **Hypermix® OIL** spray from the second week since burn occurred. Courtesy: **Dr. Alessandro Guerra**

For information write to:

info@hypermix.it

FORMULATIONS

HYPERMIX® is available in the formulation **OILY, GEL** and **MEDICATED GAUZE** to treat a variety of skin lesions.

The increased concentration of components in the **OILY formulation** helps to control infections, thus improving a cleansing and debridement action. The OILY formulation is also recommended when the GEL formulation is not able to reach the treatment area.

The **GEL formulation** has a lower concentration of the components, without any impact on the effectiveness of the product.

MEDICATED GAUZE with **HYPERMIX® GEL** are recommended in case of large lesions. Disposable, not absorbable.

HOW TO USE HYPERMIX?

Clean the skin with sterile solution.


Apply a veil of HYPERMIX® over the entire lesion with a frequency of once a day until the complete healing.

The frequency of treatment can be varied according to the type of injury at the discretion of the veterinarian.

APPLICATION

Always apply a thin layer of GEL or **HYPERMIX® OIL** to form a thin invisible barrier on the lesion.

It is recommended to apply the product beyond the edges of the wound, to ensure proper tissue regeneration. Exceeding the recommended amount does not increase the effectiveness of the product.

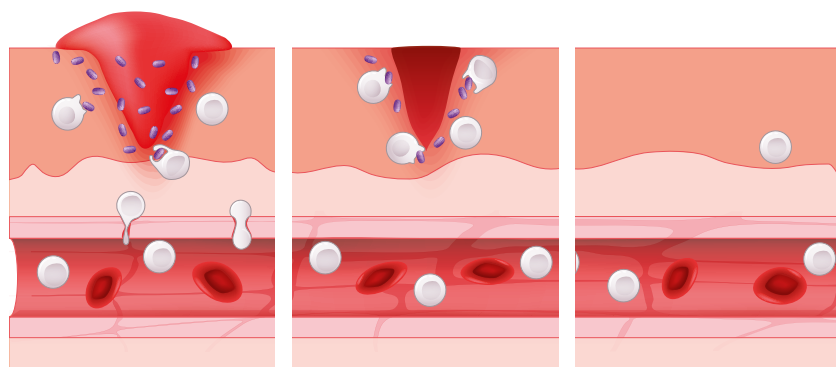

Apply the **MEDICATED GAUZE HYPERMIX®** directly on the lesion and cover the treated skin with breathable bandages or gauze.

Coverage is recommended for animals exposed to the sun usually, for very large lesions at risk of environmental contamination or in subjects living with other animals.

It is advisable to replace the medicated gauze every 24 hours or as directed by your veterinarian.

OILY



100 ml Glass Bottle
(with spray nozzle)

REF. 100600



30 ml Spray can
(dispenses 380 sprays)

REF. 10030SP



5 Vials of 5 ml each
(individually packed)

REF. 100102



50 Vials of 5ml each
(individually packed)

REF. 9000150

GEL



30 ml Tube

REF. 10030TGY



200 ml Jar

REF. 10200VA



30 Medicated Gauzes with GEL 10x10 cm
(individually packed)

REF. 10030GMS

1 Medicated Gauze with GEL 10x10 cm

REF. 10001GMS-V

KANARESCUE®

The veterinary line of RI.MOS.
also includes KANARESCUE®
Find out more on:

www.kanarescue.it



RI.MOS.
Medical Products

Via Manuzio, 15 - 41037 - Mirandola (MO) - ITALY
Tel: +39 0535 25755 - info@rimos.com - www.rimos.com

ri.mos_vet



QUALITY MANAGEMENT SYSTEM
ISO 13485:2016

COD. 100910X-ENG
Rev. 01-2025