



CAESAREAN SECTION

TRANSABDOMINAL PLACEMENT DURING C-SECTION:

CATHETER THROUGH THE HYSTEROTOMY

Tip with inflation fittings → VAGINAL CANAL

Tip with balloon → UTERINE CAVITY

BALLOONS INFLATION

1° OUTER BALLOON

60 ML syringe of sterile saline solution
(close the stopcock)

→ **BLUE LUMEN: "WATER"**

2° INTERNAL BALLOON

Same 60 ML syringe of air based on the uterine volume.

→ **TRANSPARENT LUMEN: "AIR INFLATION"**

MAX 420 CC

count out loud about 7 PUMPS CONTINUOUSLY
(close the stopcock)

DRAINAGE BAG CONNECTION

Fluid collection bag (included)

→ **RED LUMEN: "DRAINAGE"**

CATHETER REMOVAL


1° INTERNAL BALLOON

TRANSPARENT LUMEN → Remove the valve in order to leak the air out

2° OUTER BALLOON

BLUE LUMEN → Aspirate the contents with a syringe

UTERUS EVALUATION BY DIRECT EXAMINATION

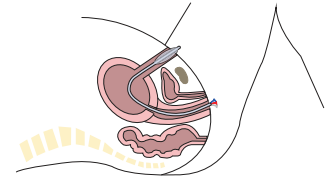

1

2




It is recommended to insert the bent tubing inside the hysterotomy, unscrewing the 3-way valve to reduce the encumbrance. Then remove the tubes, one by one, inside the vaginal canal and reconnect the 3-way valve.




It is however possible to position the device through the vaginal canal, after vaginal washing with an antiseptic solution.


3

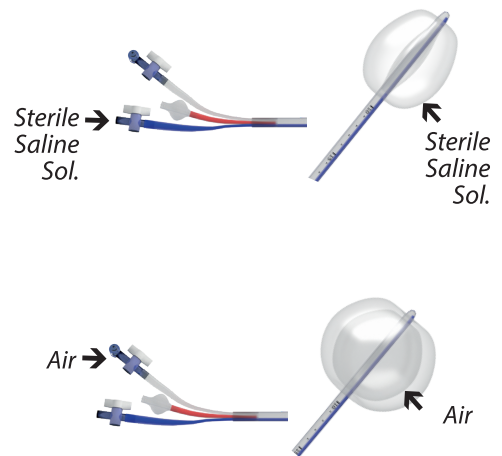

4

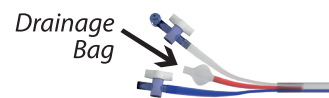

5

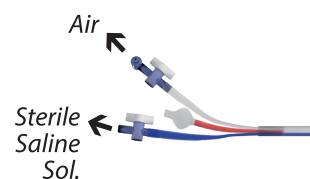

It is also possible to cut the lumens of the catheter to deflate the balloons

GENTLY RETRACT THE CATHETER FROM THE UTERUS AND CERVICAL CANAL AND WAIT SOME MINUTES TO MAKE SURE THERE IS NO RESTART OF UTERINE BLEEDING



VAGINAL BIRTH

TRANSVAGINAL PLACEMENT

TIP WITH BALLOON → UTERUS

BALLOONS INFLATION

1° OUTER BALLOON

60 ML syringe of sterile saline solution
(close the stopcock)

→ **BLUE LUMEN: "WATER"**

2° INTERNAL BALLOON

Same 60 ML syringe of air based on the uterine volume.

→ **TRANSPARENT LUMEN: "AIR INFLATION"**

MAX 420 CC

count out loud about 7 PUMPS CONTINUOUSLY
(close the stopcock)

DRAINAGE BAG CONNECTION

Fluid collection bag (included)

→ **RED LUMEN: "DRAINAGE"**

CATHETER REMOVAL

1° INTERNAL BALLOON

TRANSPARENT LUMEN → Remove the valve
in order to leak
the air out


2° OUTER BALLOON

BLUE LUMEN → Aspirate the contents
with a syringe


1

UTERUS EVALUATION
BY DIRECT EXAMINATION
AND/OR ULTRASOUND EXAMINATION

2

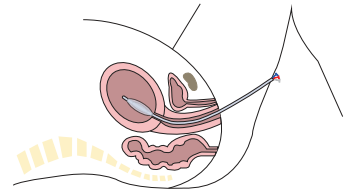


3

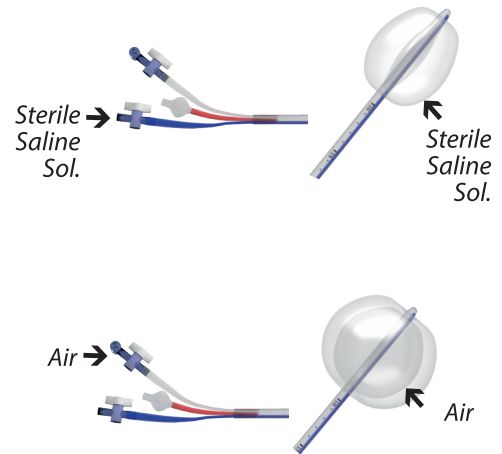

4



5

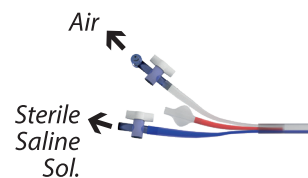

It is also possible to cut the lumens of the catheter to deflate the balloons

GENTLY RETRACT THE CATHETER FROM THE UTERUS AND CERVICAL CANAL AND WAIT SOME MINUTES TO MAKE SURE THERE IS NO RESTART OF UTERINE BLEEDING